making work easy

- [FAQ - Deutsch](#)
- [FAQ - English](#)
- [FAQ - Français](#)
- [FAQ - Italiano](#)
- [FAQ - Español](#)
- [FAQ - Português](#)
- [FAQ - Türkçe](#)
- [FAQ - Русский](#)
- [FAQ - Polski](#)
- [FAQ - Svenska](#)
- [FAQ - Čeština](#)
- [FAQ - 中文](#)
- [FAQ - 日本語](#)
- [FAQ - 한국어](#)
- [اللغة العربية - FAQ](#)
- [FAQ - Українська](#)
- [FAQ - Suomi](#)

# VORTEX compact 3L

[FAQ - frequently asked questions](#)

# FAQ - Deutsch

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# FAQ - English

*For chapter indications, refer to the complete instruction manual on the internet.*

Problem	Cause	Remedy
<b>Motor won't run.</b>	<ul style="list-style-type: none"><li>Fuse blown in mains socket.</li><li>Overload protection triggered.</li><li>Motor defective.</li></ul>	<ul style="list-style-type: none"><li>Reactivate mains fuse.</li><li>Turn off dust extractor and allow to cool down for 5 minutes. Contact the Renfert Service Team if the extractor still fails to restart.</li><li>If the extractor is to be situated in a base cabinet, ensure there is sufficient air cooling.</li><li>Have Renfert Service Team replace the motor.</li></ul>
<b>Motor won't run in Auto mode.</b>	<ul style="list-style-type: none"><li>Electrical appliance defective or not connected properly.</li><li>Low power consumption of electrical appliance.</li></ul>	<ul style="list-style-type: none"><li>Check operability of electrical appliance and check if connector is tight.</li><li>Note that the min. rated power consumption is <math>P &gt; 20 \text{ W}</math>.</li></ul>
<b>Reduced suction power.</b>	<ul style="list-style-type: none"><li>Suction control turned down too low.</li><li>Suction hose / nozzle clogged.</li><li>Disposal bag or dust bag full.</li><li>Fine particle filter dirty.</li><li>Seal between container and top of extractor housing defective or not there.</li><li>«Free-Filter-System» mechanism defective.</li><li>Activation button for the filter cleaning function is pressed due to coiled cable.</li></ul>	<ul style="list-style-type: none"><li>Set suction control as described in "Suction control adjustment".</li><li>Clean suction hose / nozzle.</li><li>See section "Replacing the disposal bag, dust bag".</li><li>See section "Cleaning the fine particle filter", "Replacing the fine particle filter".</li><li>Replace seal.</li><li>Contact Renfert Service Team.</li><li>Wind the cable up so that the activation button is not actuated.</li></ul>
<b>Extractor won't pick up wet substance.</b>	<ul style="list-style-type: none"><li>Dust retainer full.</li></ul>	<ul style="list-style-type: none"><li>Turn off unit. Empty dust retainer.</li></ul>

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