

Ultra Band-Lok® & Ultra Band-Lok® BLUE V13 3/17
Light Cure Cement Available With and Without Fluoride

INDICATIONS FOR USE:

Ultra Band-Lok® is intended for use as an orthodontic band cement and for occlusal buildup.

PRODUCT FEATURES:

Ultra Band-Lok® is also available in Blue Shade which will contrast the enamel and facilitate removal of material before and after treatment.

INSTRUCTIONS FOR USE:

1. With 1st & Final®, thoroughly prophyl the teeth to be banded. Rinse, dry and isolate. Acid etching of the enamel is not required for cementing bands, however it will increase strength required for high stress banded appliances such as headgear, lip bumpers and banded Rapid Palate Expanders.
2. With a microetcher or carbide bur, roughen the inside of the band(s) if this has not been performed by the manufacturer.
3. While holding the band between the thumb and forefinger, apply a bead of Ultra Band-Lok® to the inside of the band. Using a small spatula or similar instrument, cover the entire inside of the band with paste. Do not fill the band with paste, as this is not necessary and wasteful.

If using the Luer-Loc syringes:

- a) Unscrew and remove the black storage cap from the tip of the syringe.
- b) Snap the needle tip in place and twist clockwise until the needle is locked in position.
- c) When the procedure is completed, remove the needle tip and replace the black storage cap to keep the Ultra Band-Lok® protected from light.
- d) **SPECIAL SAFETY NOTE REGARDING SAFETY RISK TO PATIENT: LUER LOK TIPS ARE SINGLE USE DEVICES. A NEW TIP IS REQUIRED FOR EVERY PATIENT TO AVOID CONTAMINATION RISK.**

Important: Ultra Band-Lok® is sensitive to ambient light! If multiple bands are filled at once, shield those bands from any light until ready to be seated.

4. Seat band, clean off any excess flash. To facilitate flash removal, shine the light on the occlusal surface for 1 second. This will make the excess paste rubbery and easy to remove in one piece. Position light tip in close proximity to the occlusal surface of the tooth and band interface. Moving in a slow circle around the top of the band, light cure for 30 seconds. At this point the cement is fully cured, allowing for immediate placement.

Note: The curing light utilized should be capable of providing a minimum of 600 milliWatts of continuous output to cure material properly in the time frame mentioned. Check your curing light with a radiometer monthly.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.



TEMPERATURE LIMITATION: Store refrigerated (2-8 °C). Avoid excessive heat. Allow product to reach room temperature prior to use.

TO REORDER:

Syringes with Fluoride: Ref #UBLBPP for Ultra Band-Lok® Blue push syringe
Ref# UBLPP for Ultra Band-Lok® Natural push syringe
Ref# LLUBLBP for Ultra Band-Lok® Blue Shade Luer-Loc syringe
Ref# LLUBLP for Ultra Band-Lok® Natural Shade Luer-Loc syringe
Luer-Loc Tips only: Ref# LLTIPS

Syringes Non- Fluoride: Ref #NFUBLBPP for Ultra Band-Lok® Blue push syringe
Ref# NFUBLPP for Ultra Band-Lok® Natural push syringe
Ref# NFLLUBLBP for Ultra Band-Lok® Blue Shade Luer-Loc syringe
Ref# NFLLUBLP for Ultra Band-Lok® Natural Shade Luer-Loc syringe

WARRANTY: Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com



Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Scan for more information