Description

The Cavifil Injector is used to apply dental filling materials directly from the Cavifil into the tooth cavity to be restored.

Fig. 1 Cavifil Injector (1) and the Cavifil (2), which is introduced from below.

Fig. 2 The lever is uniformly activated to extrude the contents of the Cavifil.

Application:

- Introduce the Cavifil (capsule filled with restorative material) into the respective recess of the Injector from below and press until it snaps into place (Fig. 1).
- The Cavifil is now secured in the Injector and can be rotated 360°, if required.
- Apply the Cavifil content by uniformly activating the lever (Fig. 2).
- Excessively fast extrusion of the contents from the Cavifil or the exertion of extremely high force during application should be avoided.
- After applying the material, remove the Cavifil by pressing it downwards and prepare the Cavifil Injector for reuse.

Disinfection, cleaning and sterilization**General notes:**

Please also observe the country-specific regulations and guidelines regarding the hygiene standards and cleaning/disinfection of medical devices in dental practices.

New non-sterile Cavifil Injectors must be subjected to a complete preparation cycle before they are used for the first time. The Cavifil Injectors must be cleaned and disinfected after every use. Oxidizing disinfectant solutions are not suitable. Furthermore, sterilization in the steam sterilizer is explicitly recommended!

Do not use damaged Injectors. Maintenance and repair work should only be carried out by specialist staff.

Protective gloves and goggles complying with the requirements of directive 89/686/EEC must be worn to handle any used and contaminated Injectors.

Generally, automated cleaning in a washer disinfector (WD) unit is preferable to manual cleaning!

Manual cleaning should only be used if automated cleaning is not possible. Only a WD (compliant with EN ISO 15883) with proven effectiveness must be used to ensure reliable automated cleaning and disinfection. The operator is responsible for ensuring that the validation, performance re-qualification and periodic routine checks of the cleaning procedures in the washer disinfector are defined, documented and carried out appropriately.

Restrictions on cleaning and disinfecting:

The end of the product's life span is mainly determined by wear and damage through use. Frequent cleaning and disinfecting have no impact on the performance of this product.

Preparation at the treatment area:

Carefully remove large unhardened residue from the Injector with a lint-free cellulose tissue immediately after use. The Cavifil Injector may either be placed in a removal container (dry removal) or directly immersed in a combination of detergent and disinfectant (wet removal), see below, for transportation to the cleaning and sterilization area. In any case, it is advisable to transport the Injector to the cleaning area as soon as possible. Waiting periods of several hours between use and cleaning should be absolutely avoided.

Cavifil® Injector

EN Instructions for Use	FI Käyttöohjeet
– Applicator for dental materials in Cavifils	– Annostusväline Cavifil-kärkiin pakatuille hammaslääketieteellisille materiaaleille
DE Gebrauchsinformation	NO Bruksanvisning
– Applikator für zahnärztliche Materialien in Cavifils	– Applikator for tannmaterialer i Cavifil
FR Mode d'emploi	NL Gebruiksaanwijzing
– Pistolet-applicateur pour matériaux dentaires en cavifils	– Applicator voor tandheelkundige materialen in Cavifils
IT Istruzioni d'uso	EL Οδηγίες Χρήσεως
– Applicatore per materiali dentali in Cavifil	– Εφαρμογές για οδοντιατρικά υλικά σε μικροσύριγγες Cavifil
ES Instrucciones de uso	TR Kullanma Talimatları
– Aplicador para materiales dentales en cavifils	– Cavifil'lerdeki dental material için aplikatör
PT Instruções de Uso	RU Инструкция
– Aplicador para materiais dentais em Cavifils	– аппликатор для стоматологических материалов в кавифилах
SV Bruksanvisning	PL Instrukcja stosowania
– Applikationsinstrument för dentala material i cavifiller	– Podajnik do aplikacji materiałów w pojemnikach cavifil

Preparation for cleaning/disinfection:

Preliminary treatment of the instruments is strongly recommended for the automated and manual cleaning procedure in order to protect personnel against infection!

To prevent residue from drying and to ensure preliminary disinfection, place the Cavifil Injector in a disinfection bath (with sieve insert and lid) filled with aldehyde-free alkaline disinfectant (e.g. ID 212 forte 2%, Dürr Dental, for 15 min.). Make sure that the Injector is completely immersed in disinfectant and that the disinfectant is free of bubbles. The disinfectant used must be suitable for the product and tested for effectiveness (e.g. DGHM or FDA certification or CE mark). Please observe the instructions of the manufacturer of the disinfectant solution (concentration rate, temperature and contact time) to ensure that the disinfectant is used correctly.

If you use an automated cleaning method, carefully rinse the Cavifil Injector under running tap water after preliminary disinfection to prevent residue of the cleaning and disinfecting agent from reaching the WD.

Automated cleaning and disinfection:

Requirements to carry out the validated procedure:

- WD (Thermo disinfector G7882, Miele, A0-value > 3000)
- cleaning and disinfection program (Vario TD)
- detergent and disinfectant (Neodisher FA, Dr. Weigert)

Position the Cavifil Injector in the WD according to the instructions of the manufacturer. Fill the WD with the respective cleaning and disinfecting agent (e.g. Neodisher FA, Dr. Weigert) as appropriate for the intended process. Observe the instructions of the disinfectant and equipment manufacturer. Start the cleaning and disinfection program (e.g. Vario TD) validated for the particular type of unit.

Drying:

After the cleaning and disinfection program has been completed, remove the Cavifil Injector from the WD and completely dry any remaining moisture with compressed air or a clean, lint-free disposable towel.

If the Cavifil Injector has only been cleaned in the WD (without automated disinfection), thermal disinfection of the unwrapped Injector in a steam sterilizer is absolutely mandatory!

Manual cleaning and disinfection:

Requirements to carry out the validated procedure:

- soft brush (Miltex nylon cleaning brush for instruments)
- detergent and disinfectant (e.g. immersion in ID 212 forte 2%, Dürr Dental, for at least 15 min.)

Use only soft cleaning brushes for the manual removal of visible residue. Never use a metal brush or steel wool. Brush the Cavifil Injector with a suitable cleaning brush (e.g. Miltex nylon cleaning brush for instruments) after immersion in the disinfection bath for 15 min. during preliminary treatment until you can no longer see any contamination on the surface. Subsequently, leave the Cavifil in the sieve insert and carefully rinse the Injector under running tap water (< 40 °C / 104 °F) until all residue is rinsed off. Do not use any chemicals for this step. Allow the remaining water to sufficiently drain off.

Place the Cavifil Injector in a disinfection bath (with a lid and sieve insert) again, filled with aldehyde-free alkaline disinfectant (e.g. ID 212 forte 2%, Dürr Dental, for 15 min.), and disinfect the Injector according to the pretreatment procedure. Make sure that the Injector is completely immersed in disinfectant and that the disinfectant is free of bubbles. Subsequently, leave the Cavifil Injector in the sieve insert and carefully rinse the Injector under running tap water (< 40 °C / 104 °F). Allow the remaining water to sufficiently drain off.

Drying:

Then, completely dry the Cavifil Injector with compressed air or a clean, lint-free disposable towel.

Thermal disinfection of the unwrapped Injector in a steam sterilizer is strongly recommended after manual cleaning and disinfection.

Maintenance, control and inspection:

After cleaning and disinfecting the Cavifil Injector, visually inspect it to see if it is clean, undamaged and functions properly. If macroscopically visible residual contamination is present, repeat the cleaning and disinfection process.

Sterilization:

Requirements to carry out the validated procedure:

- suitable sterile packaging (aluminium sterilization container)
- steam sterilizer (Systec HX-320)
- program: 3 x fractionated pre-vacuum, 132 °C (270 °F), 20 min.

It is strongly recommended to sterilize semi-critical medical devices of Classes A & B!

Packaging:

The Cavifil Injector should be autoclaved in an appropriate sterile packaging (e.g. aluminium sterilization container).

Sterilize the items in a steam sterilizer (the unit should be compliant with EN 285 and validated according to EN ISO 17665) using a fractionated pre-vacuum at 132 °C (270 °F) for at least 20 minutes. Please also observe the instructions of the equipment manufacturer.

If you sterilize several instruments at the same time in the steam sterilizer, make sure to observe the instructions of the equipment manufacturer regarding the maximum permissible load to avoid overloading. The stipulated sterilization procedure has been validated by the manufacturer as appropriate for the purpose in question according to EN ISO 17665. The manufacturer does not assume responsibility for other sterilization procedures. If other sterilization methods are used, the operator is responsible for validating them to ensure that they are effective and fit for the purpose.

Storage and shelf life:

- Store the Cavifil Injector in dry conditions to avoid condensation build-up.

Notes:

- The material content of the Cavifils should be processed at room temperature. Cold material may be difficult to extrude.
-  If the Cavifil material is directly applied from the Cavifil in the mouth of the patient, the Cavifil should only be used for one patient for reasons of hygiene. It must not be cleaned, disinfected, sterilized and reused (to prevent cross-contamination between patients).
- The Cavifil Injector is optimally coordinated with the requirements of Ivoclar Vivadent Cavifils. However, it can also be used with other suitable capsule systems.
- Since small parts generally involve a risk of ingestion and aspiration, it is strongly recommended to use a rubber dam.
-  The Cavifil Injector is autoclavable and designed for multiple use and has the ability to undergo at least 200 autoclaving cycles. For conditions see "Disinfection, cleaning and sterilization".
-  The Instructions for Use are available on the website of Ivoclar Vivadent AG (www.ivoclarvivadent.com).

Keep out of the reach of children!

For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.