3M ESPE

Impregnum® Penta™ Soft

Polyether Impression Material for 3M ESPE Pentamix™

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3M ESPE, Impregnum, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada.
© 2014, 3M. All rights reserved.

3M ESPE

ENGLISH

Product Description
Impregnum® Penta™ Soft is a medium-bodied consistency polyether impression material for the Pentamix™ mixing device. The mixing ratio is 5 parts base paste : 1 part catalyst. Due to its lower Shore hardness, the impression material is suitable for use with impressions with slight undercuts even without an additional blocking out.

These Instructions for Use should not be discarded for the duration of product use. For details on Pentamix and accessories, Polyether Adhesive, and Penta™ Elastomer Syringe, please refer to the corresponding Instructions for Use.

Indications

- Impression of inlay, onlay, crown, bridge, and veneer preparations
- Functional Impressions
- Fitment impressions
- Implant impressions

Precautionary Measures
If an adverse reaction is observed in a patient, discontinue use of the product in this patient. 3M MSDS can be obtained from www.mmm.com, or contact your local subsidiary.

Preparation

Blocking out
► Block out undercuts or areas where gingival recession exists to prevent the impression material from “locking” onto the tooth structure. Failure to block out may make tray removal difficult or cause extraction of natural teeth or prostheses. Blocking out is especially important when using impression materials with high Shore hardness.

Prepare impression trays
Impressions used for precision impressions are suitable.
► For sufficient adhesion, apply a thin layer of Polyether Adhesive to the tray and allow to dry completely (at least 30–60 sec – 15 min drying time are optimal).

Prepare Pentamix/Penta cartridge/poly bag

► Place Impregnum Penta Soft poly bags in the designated 3M ESPE Penta cartridge only!

- Place the cartridge in the Pentamix mixing unit and attach a new Penta Mixing Tip – Red.
- If a new mixing tip is installed when the cartridge is inserted, check whether the drive shaft is engaged before you start mixing.
- With newly filled cartridges, start mixing and extrude and discard the first 15 mm of the first use of the impression material. Do not use the paste to take an impression until the color of the paste is homogeneous.

Retraction

Retraction cords as well as solutions and pastes based on aluminum chloride, aluminum sulfate, or iron sulfate are suitable retraction agents.

Recommendation: 3M ESPE Astrington Retraction Paste/Retraction Capsule. The paste containing aluminum chloride is directly applied from the retraction capsule into the sulcus. The paste can be used as an alternative to, or in combination with, retraction cords and retraction methods depending on the clinical situation and the practitioner's application technique.

- Areas from which impressions are to be taken should be kept dry.
- Prior to taking the impression, completely remove all residual retraction agents by rinsing and drying.

Dosing and Mixing

- Dosing and mixing are performed automatically in the Pentamix.

Processing time	Setting time	Residence time in the mouth
02:45	from start of mixing*	min. 03:15
	06:00	03:15

The working times shown are in conformity with ISO 4823:2000. Higher temperatures will shorten the working time, while lower temperatures will prolong it. We recommend a working time of 01:45 min:sec and compliance with the indicated setting time from the start of mixing to assure an ideal impression result at all times under clinical conditions.
* Start of mixing = entry of paste into the mixing tip.

Impression Taking

► After application around the preparation, attach the Penta Elastomer Syringe to the mixing tip of the Pentamix device and fill the syringe.

► Then load the tray previously prepared with adhesive. The mixing tip should be kept immersed in the material at all times.

► Apply the material into the sulcus or cavity from bottom up, keeping the tip of the application nozzle immersed in the material and in contact with the tooth surface at all times.
– Depending on the number of units to be syringed, initiate application around the preparation appropriately to finish syringing and tray lifting at the same time. Failing this, the syringe material will set more rapidly than the tray material, leading to distortion of the impression.

► Immediately after application around the preparation, position the loaded tray in the mouth and hold it there without applying pressure until the setting is complete.

- To remove initial adhesion (“setting the valve”), especially with upper jaw impressions, remove the tray from the gingiva on one side in a posterior position. If this proves difficult, it may be necessary to carefully blow some air or water between the impression and the gingiva.

Functional impression

► After preparing the custom tray with adhesive, load the tray with paste mix, position it in the mouth, and ask the patient to perform functional movements.

► If required, use a light-bodied consistency 3M ESPE polyether impression material for precise impression taking (correction).

After impression taking

► Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual impression material from the mouth.

Hygiene

► Place the impression in a standard disinfectant solution commonly used for impressions for the period of time recommended by the manufacturer. Excessive disinfection may damage the impression.

► After disinfection, rinse the impression under running water for approx. 15 sec.

► Prepare a cast from the impression with a large size stone plaster no earlier than 30 min and no later than 14 days after impression taking.

► To avoid introducing bubbles into the model, briefly pre-rinse with water and dry with air before pouring. Do not use surfactants as these impair the quality of polyether impressions and are not necessary!

► Polyether impressions can be silver-coated, whereas copper-coating is not feasible.

Cleaning

► Paste that has not set may be removed with ethanol or by rinsing with water and soap.

Notes

► Temperatures below 18° C/64° F: The viscosity of the pastes may increase sufficiently to make mixing in the device difficult. After keeping the pastes at 18° C/64° F for at least one day, the processability is re-established without compromising quality.

- Direct exposure to sunlight and damp storage conditions may damage the impression.
- Polyether impressions should not be exposed to solvent-containing liquids as this may result in swelling and imprecise molding.
- 3M ESPE polyether materials may only be combined with 3M ESPE polyethers. The working and setting times of the individual products must be strictly observed.

Storage and Stability

Store the product at 18–25° C/64–77° F. Do not refrigerate! Do not use after the expiration date.

Store impressions dry and below 30° C/86° F in the dark.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. USer is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether such damage is direct, indirect, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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