ITALIANO

CE
0434

NON GAMMA 2 ALLOY

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CE
0434

NON GAMMA 2 ALLOY

In proximal or occlusal contact to dissimilar metal restorations. Amalgama Bestdent non Gamma 2 alloy has a high copper content, is zinc free and meets the requirements of ISO 24234. The micro-granulometry of irregular atomized and lathe cut particles and the homogeneity of the relation between the metals employed result in an unmatched plasticity during the working process, while maintaining, at the same time, its special characteristics. The silver-copper percentage and the optimum relation between alloy and mercury guarantee the utmost resistance against corrosion and breakdowns, while preserving its perfect margins.

SIDE EFFECTS:

For professional use only

The contact with mercury can lead to skin, eye, respiratory system, and mucous membranes irritations. In sporadic cases, hyper-sensitivity reactions, allergies or local troubles due to electro-chemical reactions may result.

Mercury may be sensitizing for skin and lungs, as well as act as nephrotoxin and neurotoxin.

Through electro-chemical processes, Lichen planus may be caused on mucous membranes.

After the filling or removal of filling with amalgam, a slight increase in mercury concentration in blood and urine can be detected.

MIXING TIME:

Residual mercury amalgam from filling or removed fillings should be stored in water or fixing solution in an airtight closed container. Dispose of used capsules in accordance with national regulations.

SAFETY WARNINGS

Inpiration of mercury vapours by the personnel of the dental cabinet can be avoided using masks and well aerating the room.

Avoid contact with not hardened material. Wear protection goggles and gloves.

Spillages of mercury should be removed immediately, including from places which are difficult to access. Use a plastic syringe to draw it up. Smaller quantities can be covered by sulphur powder and removed. Avoid inhalation of the vapour.

If moisture is introduced into the amalgam before it has set, properties such as strength and corrosion resistance may be affected adversely.

If the alloy contains zinc, such contamination may result in an excessive expansion (delayed expansion) whenever it is possible, use dry field.

In case of accident or if you feel unwell, seek medical advice immediately. (Show the label where possible)

The carried information meet the requirements of ISO 13897/ISO24234 and European laws

COMPOSITION:

Ag40,3%, Sn30,7%, Cu29%

Length of capsule: 33,7 mm

Exterior diameter of capsule: 14 mm

For use in dentistry only!

R50/53

ESPAÑOL

CE
0434

NON GAMMA 2 ALLOY

Dental amalgam in self activating capsules blend of irregular atomized and lathe cut particles. Amalgama Bestdent non Gamma 2 alloy has a high copper content, is zinc free and meets the requirements of ISO 24234. The micro-granulometry of irregular atomized and lathe cut particles and the homogeneity of the relation between the metals employed result in an unmatched plasticity during the working process, while maintaining, at the same time, its special characteristics. The silver-copper percentage and the optimum relation between alloy and mercury guarantee the utmost resistance against corrosion and breakdowns, while preserving its perfect margins.

SIDE EFFECTS:

For professional use only

The contact with mercury can lead to skin, eye, respiratory system, and mucous membranes irritations. In sporadic cases, hyper-sensitivity reactions, allergies or local troubles due to electro-chemical reactions may result.

Mercury may be sensitizing for skin and lungs, as well as act as nephrotoxin and neurotoxin.

Through electro-chemical processes, Lichen planus may be caused on mucous membranes.

After the filling or removal of filling with amalgam, a slight increase in mercury concentration in blood and urine can be detected.

MIXING TIME:

Residual mercury amalgam from filling or removed fillings should be stored in water or fixing solution in an airtight closed container. Dispose of used capsules in accordance with national regulations.

SAFETY WARNINGS

Inpiration of mercury vapours by the personnel of the dental cabinet can be avoided using masks and well aerating the room.

Avoid contact with not hardened material. Wear protection goggles and gloves.

Spillages of mercury should be removed immediately, including from places which are difficult to access. Use a plastic syringe to draw it up. Smaller quantities can be covered by sulphur powder and removed. Avoid inhalation of the vapour.

If moisture is introduced into the amalgam before it has set, properties such as strength and corrosion resistance may be affected adversely.

If the alloy contains zinc, such contamination may result in an excessive expansion (delayed expansion) whenever it is possible, use dry field.

In case of accident or if you feel unwell, seek medical advice immediately. (Show the label where possible)

The carried information meet the requirements of ISO 13897 / ISO 24234 and European laws

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