Komet Dental
Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25
32657 Lemgo · Germany
Telefon +49 (0) 5261 701-0
Telefax +49 (0) 5261 701-289
info@kometdental.de
www.kometdental.de

OS-Scheiben
© 03/2016, Gebr. Brasseler GmbH & Co. KG
303964 - Rev 2016-03

DE **EN** **FR** **ES** **IT**

KFO, Orthodontics, Traitement orthodontique,
Gnato – ortopedia, Ortodonzia

DE

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EN

Oscillating discs for interproximal enamel reduction (IPR)

1. Indication
Interproximal reduction of enamel within the course of an orthodontic treatment.

2. ContraindicationTreatments during which the instruments might jam. Do not use as a lever.

3. Recommendations for use
- According to the amount of substance to be removed, choose segment discs with medium diamond grit for high substance removal (identification: blue ring) or segment discs with fine diamond grit for lower substance removal (identification: red ring).
- The respective thickness of the disc is laser marked on the shank.
- The discs with one-sided diamond coating are especially suited for stripping one proximal surface only (“...V” means coated on the upper side and “...H” means coated on the lower side”).
- The discs are to be used in the Komet oscillating contra-angle OS30 only. The interior partition of the contra-angle is at 45°. The instruments can be chucked in different positions to suit the given circumstances. Ensure that the instrument is securely locked in place.
- The instrument should rotate at the maximum engine setting of ⊖_{max} 40.000 rpm when applied to the tooth. Cool with spray.
- In case you are using an air motor, work at the maximum engine power of ⊖_{max} 20.000 rpm
- The optimum oscillating speed (see label or accompanying information in sets) is reached by setting the maximum engine power.
- Separate the interproximal contact point with diamond strips. Start with thinner and follow with thicker strips, if necessary. Following that, apply the oscillating disc in occlusal or vestibular position and guide slowly, yet steadily, through the contact point.
- Strip the tooth to be prepared to the required width.
- Do not exceed the maximum oscillating speed indicated on the package. Risk of injury through instrument fracture!
- Always supply plenty of spray cooling (at least 50 ml/min.)!
- Avoid excess contact pressure which might bend the disc.
- Do not allow the discs to come in contact with soft tissue.
- A final fluoridation is recommended.
Information on the IPR set 4594
- Eliminate the interdental contact using the diamond strips with honeycomb design (first yellow, then blue)
- Determine the amount of substance to be removed according to individual requirements and the IPR plan (reproximation chart).
- The segment discs within the 3 columns (IPR target reduction 0,2, 0,3 and 0,4 mm) are to be used one after the other up to the last disc within the corresponding section of the required enamel reduction. The segment discs with one-sided diamond coating allow for selective reduction on one proximal surface only.
- Subsequent to eliminating the interproximal contact, an enamel reduction of 0.5 mm can be achieved directly using the rotary instrument 8392 (red ring), which is guided from horizontal into the interproximal space.

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Instructions for use

Please read carefully and retain for future reference.

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- Diamond instrument 850 is used to round off and recontour the proximal edges and incisal surfaces.
- The segment discs in the individual sections are undersized by 0.05 mm so that the exact target reduction is achieved with final polishing.
- The required target reduction of the individual sections (IPR target reduction) is therefore achieved by final polishing.

4. Cleaning, disinfection and sterilisation
The instrument is to be treated with anticorrosive disinfecting and cleaning agents for rotary instruments (e.g. Komet DC1). Regarding recommendations for use (immersion time, concentration, suitability) of the disinfecting or cleaning agents please refer to the instructions provided by the manufacturers. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instrument in wet or humid condition for a prolonged period of time. Make sure that instruments do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfecter, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Inspect cleaned instruments visually. Any damaged or blunt separating strips are to be rejected and discarded. Sterilization is carried out in the autoclave at 134 °C. Please observe manufacturers’ instructions of these devices. Corroded instruments must be rejected and their use discontinued. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Information on proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

Instructions with regard to the IPR set 4594

Having used the set 4594, please make sure that the single components of the set (i. e. the insert tray containing the instruments and the sterilization container) are placed separately into the thermodisinfecter for mechanical reprocessing. The insert tray can be placed back into the sterilization container when sterilised in the autoclave. Please refer to the instruction for use that come with the sterilization containers.

5. Safety and liability

Worn and damaged (i.e. bent etc.) instruments are to be discarded and immediately replaced by new ones. The above instructions regarding the handling of the instruments, appropriate cooling and contact pressure are to be strictly observed. Only use the instruments according to their intended purpose. The non-observance of these safety instructions can lead to damage to the power system or injuries. It is the responsibility of the user to check prior to use that the instruments are suitable for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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