VOCO

Bifix® Veneer Try-In

CE 0482

EN Instructions for use
MD EU Medical device

Product description:

Bifix Veneer Try-In is a water-soluble, coloured glycerine paste.

The shade appearance of the cured **Bifix Veneer LC** can be simulated by inserting the permanent restoration with **Bifix Veneer Try-In** in the selected shade. The final shade appearance is a mixture of natural tooth shade, the shade of the restoration being luted and the luting material itself.

Shades:

Bright, Clear, Warm

Indications:

For selection of the shade of **Bifix Veneer LC** for the subsequent permanent luting of the restoration.

Contraindications:

No contraindications are known.

Patient target group:

Bifix Veneer Try-In is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use.

User:

Bifix Veneer Try-In should only be used by a professionally trained dental practitioner.

Use:

Remove the temporary restoration, if present. Clean the tooth, rinse thoroughly and dry gently with air. Avoid overdrying. Lock the supplied type 50 application tip on the syringe by turning it clockwise and apply **Bifix Veneer Try-In** to the adhesive surface of the restoration before carefully positioning it on the tooth. The attached application tip is intended for single use only. Remove it after use and seal the syringe again tightly with its original cap. Protect the syringe from contamination. Following the shade check, remove the restoration and rinse the **Bifix Veneer Try-In** off the tooth and restoration thoroughly with a water spray, and then dry both. Take care not to overdry the dentine. Any residual **Bifix Veneer Try-In** can affect the subsequent permanent luting of the restoration.

Warnings, precautionary measures:

- The shades of the try-in pastes correspond to those of the cured luting material. They therefore differ slightly in shade from the uncured material, which only acquires its final shade after polymerisation.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Storage at 4°C-28°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Bifix Veneer Try-In** must be reported to VOCO GmbH and the responsible authority.

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