FICHA TÉCNICA

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TECHNICAL DATA SHEET

DRILL DISINFECTION BESTDENT		
REFERENCE	60155	
MANUFACTURER	PRISMAN GmbH Otto-Hahn-Ring 6-18 D-64653 Lorsch Phone: +49 (0)6251-866 980-0	
CLASSIFICATION	Class IIb Medical Device, Directive 93/42/EEC.	
DESCRIPTION	- Anticorrosive - Excellent material compatibility. - Its mild ph-value preserves the abrasiveness of cutting edges. - Broad spectrum of activity, excellent cleaning properties.	
COMPOSITION	Contains < 5% non-ionic surfactants, <5% cationic surfactants surfactants, <5% EDTA salts, fragrance.	
PACKAGE	2 liter drum.	
INTENDED USE	Used for cleaning and disinfecting of drills and rotating dental instruments.	
INSTRUCTIONS FOR USE	See product instructions. Microbiological Effectiveness: Bactericidal in accordance to VAH/DGHM, EN 13727, EN 14561. Levurocidal (Candida albicans) in accordance to VAH/DGHM, EN 13624, EN 14562. Tuberculocidal in accordance to VAH/DGHM, membrane covered viruses (HBV, HIV, HCV) in accordance to BGA(RKI)/DVV. Effectivity: Bactericidal in accordance to VAH/DGHM, EN 13727, EN 14561 (high bio load): Conc.: 15min. Levurocidal in accordance to VAH/DGHM, EN 13624, EN 14652 (high bio load): Conc.: 15min. TbB (Tuberculosis)-Inactivation in accordance to VAH/DGHM: Conc.: 30min. Membrane covered viruses (i.e. HBV, HIV, HCV) in accordance to BGA(RKI)/DVV with bio load: Conc.: 15min.	
INDICATIONS FOR USE	Drill Bath is a ready to use solution. Place the used instruments into the solution, completely submerged. Once the resting time is over, rinse instruments with water and dry. Do not mix with other disinfectants or cleansers. Not suitable for Aluminum.	
STORAGE AND TRANSPORT	Store in a cool and dry place. Keep away from direct sunlight and heat sources. Protect from humidity.	