FICHA TÉCNICA

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TECHNICAL DATA SHEET

PROCLINIC TEST BOWIE & DICK		
REFERENCES	79955	
MANUFACTURER	E.C.S. S.r.l. Via Como, 71, 23883 Brivio (LC), Italia info@ecssrl.com – www.ecssrl.com	
DESCRIPTION	Disposable test to verify the correct functioning of the vacuum system in type B steam autoclaves. Simulates a porous load and detects possible failures in air removal and steam penetration. Complies with EN ISO 11140-1, 11140-3, and 11140-4.	
SPECIFICATION	- Compatible with steam sterilization cycles: - 134°C for 3.5 minutes - 121°C for 15 minutes - The indicator changes from yellow to grey/black if the cycle is correct. - Clear visual reading: pass/fail. - Single-use and disposable.	
COMPOSITION	- External packaging: cardboard box with process indicator. - Interior: two filtering layers of pure pulp paper and a central card with a chemical indicator.	
PACKAGING	Box with 1 unit (individual test).	
INTENDED USE	Daily control of the correct functioning of the vacuum system in type B steam autoclaves. Facilitates traceability and patient safety.	
SHELF LIFE	5 years from the date of manufacture, if storage conditions are respected.	
INSTRUCTIONS FOR USE	1. Ensure the autoclave has completed the warming cycle. 2. Place the test in the center of the bottom shelf of the autoclave, indicator facing up. 3. Run the sterilization cycle. 4. Remove the test with care (it may be hot). 5. Remove the indicator card and check the result: - Pass: complete color change to grey/black. - Fail: incomplete or different color change.	
HANDLING AND STORAGE CONDITIONS	- Store away from light and heat sources. - Avoid polluted environments. - Do not store directly on walls or floors. - Use in order of production. - Handle with care. - Failure to comply with these conditions may affect product performance.	