

# Operating Instructions

Status of technical operating instructions 1/96

---

## **1. General terms:**

**1.1. Product description:** CUTEX Undercut wax

**1.2. Delivery form:** tin of 20g / box of 540 g

**1.3. Manufacturer:** YETI Dentalprodukte GmbH - Industriestraße 3  
78234 Engen/Germany  
Tel.: 0049-7733-9410-0 Fax: 0049-7733-94 10 22

**1.4. Storage:** Store at a temperature of at least +10°C to 30°C  
Protect from UV-radiation; temperatures below 8°C,  
especially below 0°C make the wax brittle. Protect  
from blow or any impacts.

## **2. Application according to determinations:**

Our different waxes are used for sculpturing of tooth restoration in dental laboratories in wax as an initial stage for the final denture. Due to the variety of colours and different characteristics, these waxes allows to practice all the different techniques.

**Any claims on warranty or liability will not be accepted if not used according to determinations !**

## **3. Risk and danger during application:**

1. Ensure that unqualified persons do not come into contact with this product.
2. Avoid hot wax drops on skin and avoid also inhalation of wax vapours

## **4. Operating instructions:**

### **4.1. Preparation**

Open the tin.

### **4.2. Application**

Choose a suitable instrument or an electric waxer for wax sculpturing. The object you want to wax up has to be isolated. Use Yeti Lube as separating liquid. Then begin your waxing-up. Surplus wax can be scraped off.

### **4.3 After application**

Close the tin after use to protect from dust or any other pollution.

**5. Disposal:** According to current local regulations.

**6. Durability:** Consumer goods should be used rapid after opening and used up within a few months.