Dental adhesive

TOTAL ETCH

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ENGLISH:

1. Isolate and prepare tooth. 2. Apply Etch as per manufacturer's instructions. Wash thoroughly. Remove excess water. Keep moist. 3. Apply Dental Adhesive Total Etch to saturate all internal surfaces. 4. Blow gently with dry, oil free air for 2 seconds to evaporate solvent. Leave surface glossy. 5. Light cure for 10 seconds. 6. Apply composite as per manufacturer's instructions. Precautions: 1. Avoid prolonged contact with skin / soft tissues as it may cause inflammation / sensitisation. 2. Discontinue use if irritation occurs or if have known acrylic resin allergies. 3. Use at room temperature. 4. When not in use, replace cap.

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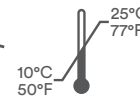
COMPOSITION:

Acrylic monomer, Acetone, Fluoride, Stabilizer.

INDICATIONS: I) All direct restorative composites. II) Compomers. III) Composite / Ceramic* / Metal* / Porcelain* repairs. *Bonding surface requires pre-treatment with a silane primer.

FIRST AID:

Eye: Immediately flush eyes with water for at least 15 minutes. Seek medical attention. Skin: Wash thoroughly with water. If irritation (redness, rash, blistering) develops, seek medical attention. Ingestion: Drink plenty of water/milk. Seek medical attention. Inhalation: No symptoms expected.

CAUTION: KEEP OUT OF REACH OF CHILDREN. FOR USE BY DENTISTS ONLY.

Full instructions and more information available at www.proclinic-products.com

CHL MEDICAL SOLUTIONS, SRL
Via Felice Cavallotti, 15
20122 Milano (Italy)