# Nano dental adhesive

## SELF-ETCH

1 bottle  
5 ml

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#### ENGLISH:

1. Isolate and prepare tooth. 2. Apply Proclinic Expert Self-etch Dental Adhesive to saturate all internal surfaces and leave in place for 20 seconds. 3. Blow firmly with dry, oil free air for 5 seconds. Leave surface glossy. 4. Light cure for 10 seconds. 5. Apply composite as per manufacturer's instructions. Precautions: 1. Avoid prolonged contact with skin / soft tissues as it may cause inflammation / sensitisation. 2. Discontinue use if irritation occurs or if have known acrylic resin allergies. 3. Use at room temperature. 4. When not in use, replace cap.

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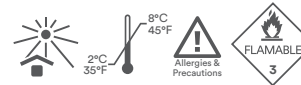
#### COMPOSITION:

Phosphoric acid ester monomer, Dimethacrylate monomer, Monomethacrylate monomer, Silicon dioxide filler, Water, Acetone, Photoinitiators, Stabilizer, Sodium Fluoride.

INDICATIONS: I) Direct filling restorations using light-cured composite and compomers. II) Composite repair of porcelain and hybrid ceramic restorations. III) Sealing exposed root surfaces causing hypersensitivity. IV) Cavity sealing prior to placement of indirect restorations. V) Core build-ups using light cure or dual cure composites. VI) Composite repair.

#### FIRST AID:

Eye: Immediately flush eyes with water for at least 15 minutes. Seek medical attention. Skin: Wash thoroughly with water. If irritation (redness, rash, blistering) develops, seek medical attention. Ingestion: Drink plenty of water/milk. Seek medical attention. Inhalation: No adverse reaction is expected for a small quantity. If signs / symptoms develop, seek medical attention.

CAUTION: KEEP OUT OF REACH OF CHILDREN. FOR USE BY DENTISTS ONLY.

KEEP REFRIGERATED UNTIL USE. Allow product to reach room temperature just before use.

Full instructions and more information available at [www.proclinic-products.com](http://www.proclinic-products.com)

CHL MEDICAL SOLUTIONS, SRL  
Via Felice Cavallotti, 15  
20122 Milano (Italy)

CE  
0476