Root canal posts

Instructions for use

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EN Instructions for use.

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DE

GLASFASER PROCLINIC

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EN

PROCLINIC GLASS FIBRE POSTS

1. Remove the guttapercha with the usual instruments.
2. Choose the post size according to clinical evaluation and x-ray.
Only for sterile posts. Do not use if the blister is damaged.
3. Determine the post length (remember the guttapercha seal in the 1/3 apex).
4. Get the root canal ready with the appropriate calibrated drill.
5. Try the post in the root canal to determine the length.
6. Cut the post at a low speed with a diamond cutting disc. Never use tweezers as they will cut the fibre in the post.
7. Clean the post surface with alcohol to remove impurities.
8. Etch the root with the etching gel.
9. Rinse the root canal well, taking care to remove all the etching (use of an endodontic irrigation needle is advised).
10. Dry the root canal with paper cones, leaving the surfaces damp.
11. Apply the adhesive in the root canal (according to manufacturers instructions).
12. Light cure the adhesive.
13. Apply the adhesive on the post.
14. Light cure the adhesive.
15. Inject the cement into the root canal.
16. Insert the post in the root canal, pushing it gently, spread the surplus cement around

the post and keep it still until completely hardened.

17. Only for glass fibre post; light cure for a minimum of 60°.

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AVENIR S.r.l.