Dental adhesive

UNIVERSAL

Instructions for use

DE Gebrauchsanweisung.
EN Instructions for use.
ES Instrucciones de uso.
FR Mode d'emploi.
IT Istruzioni per l'uso.
PL Instrukcja postępowania.
PT Instruções de uso.

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UNIVERSAL DENTAL ADHESIVE

Definition

Universal dental adhesive is a light-cured, one-component bonding agent used with the self-etching, selectively etching or total etch technique for adhesive restoration techniques.

Composition:

Methacrylates
Photoinitiators
Ethanol
Water

Areas of application

- Bonding in adhesive restoration techniques
- Adhesive attachment of composite materials and composites on enamel and dentine
- Adhesive attachment of ceramic and composite restorations on enamel and dentine
- Composite material on ceramic, composite material, metal and amalgam
- Dentine sealing

Application techniques

- Self-Etch: Universal dental adhesive is applied on enamel and dentine
- Selective-Etch: Enamel is etched, Universal dental adhesive is applied on enamel and dentine
- Total-Etch: Enamel and dentine is etched, Universal dental adhesive is applied on enamel and dentine

Contraindications

Proven hypersensitivities to ingredients contained in Universal dental adhesive. Inadequate oral hygiene. If it is impossible to keep the working area dry.

Side effects

Universal dental adhesive ingredients can cause sensitivity in predisposed persons.

Interactions

Phenolic and other substances (e.g. zinc oxide eugenol) which inhibit polymerisation must not be allowed to come into contact with Universal dental adhesive.

Applications

Note
To protect against the bond curing, replace the cover on the mixing well immediately after dispensing.

1. Conventional filling technique

For self-etching previous etching with etching gel is not necessary. For etching with etching gel we advise you to follow the following technique:

1.1 Preparing the cavity

According to the principle of the adhesive restoration technique. If indicated, a punctiform cap should be placed near the pulp using hard curing calcium hydroxide cements. Finish the enamel with a diamond bur. In case of uncut enamel, etch with phosphoric acid or clean with pumice. Then thoroughly rinse and dry the enamel. The use of a rubber dam is recommended for hygienic reasons.

1.2 Conditioning

Apply phosphoric acid based etchant gel in accordance with the manufacturer's instructions for use with
a) Selective enamel etching on enamel
b) Total-Etch on enamel and dentine
Dry the cavity with a foam pellet or with oil-free compressed air. Important: Do not let dentine dry out.

1.3 Application From the bottle:

Shake the bottle well before use and dispense a drop into the mixing well. Close the bottle again immediately. Rub Universal dental adhesive into the cavity with a disposable dentalbrush for 20 s. Blow gently with oil-free compressed air for 5 s. Then light cure for 10 s (halogen or LED polymerization unit with light intensity >800mW/cm²).
At lower light intensity the light curing is prolonged accordingly.

From the single dose:

Open the single dose immediately before use. Snap off the safety lock, rub Universal dental adhesive into the cavity with a disposable dentalbrush for 20 s. Blow gently with oil-free compressed air for 5 s. Then light cure for 10 s (halogen or LED polymerization unit with light intensity >800mW/cm²). At lower light intensity the light curing is prolonged accordingly. Avoid adhesives, as they could impair the accuracy of fit.

1.4 Composite/composer filling material

Use the restoration material according to the manufacturer's specifications.

2. Fixation of inlays/onlays, veneers and crowns

The use of Universal dental adhesive is only possible under the condition that sufficient incident light (according to the manufacturer's specifications) through the complete restoration is ensured.

2.1 Preparing the cavity Proceed as in point 1.1

2.2 Conditioning Proceed as in point 1.2

2.3 Application Proceed as in point 1.3

2.4 Cementing

According to the respective manufacturer's specifications. Important: Avoid over application of the adhesive as this could impact the fit of the restoration.

3. Bonding to other surfaces

3.1 Ceramics

Roughen the ceramic surface with a diamond bur or sand-blast, clean the ceramic surface with phosphoric acid, rinse and blow with oil-free compressed air.

3.2 Metal and amalgam

Roughen the surface with a diamond bur or sand-blast. Then clean with a steam jet (extraoral) or acetone and dry with oil-free compressed air.

3.3 Composite

Roughen the composite surface with a diamond bur or sand-blast, clean with phosphoric acid, rinse and dry with oil-free compressed air.

3.4 Application Proceed as in point 1.3

4. Sealing dentine

Proceed as in point 1.1, 1.2 and 1.3

Emergency measures, symptoms, antidotes

If the product comes into contact with the oral mucosa, rinsing with water is sufficient. If the product comes into contact with eyes, rinse thoroughly with water. Consult an ophthalmologist for assessment and further treatment.

Note

Product is only to be provided to dentists and dental technicians or on their behalf. Keep out of the reach of children. Only dispose of completely emptied packages.

Expiry date and labelling

The expiry date and code are found on the containers and external packaging.
Do not use the product beyond its expiry date.

Storage

4-8 °C / 39-46 °F
Do not expose to direct sunlight or other heat sources.

Hygiene tips

Observing the following instructions ensures optimal hygienic conditions for the use of Universal dental adhesive:
• Disposable brush and single dose use for one patient only.
• Close the bottle with the sealing cap after use and disinfect with standard disinfectants.

Caution

Federal law restricts this device to sale by or on the order of a dentist.

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