RELIANCE PLASTIC APPLIANCE CONDITIONER

INSTRUCTIONS FOR USE

RELIANCE®

ORTHODONTIC
PRODUCTS

RELIANCEORTHODONTICS.COM

V1.11.20

RELIANCE PLASTIC APPLIANCE CONDITIONER

ENGLISH TRANSLATION

INDICATIONS FOR USE: The Reliance Plastic Appliance Conditioner is intended for use as a conditioner of plastic brackets/acrylic appliances and composite restorations

INSTRUCTIONS FOR USE:

BONDING A PLASTIC BRACKET OR ACRYLIC APPLIANCE:

With a brush or sponge pellet, apply a thin uniform coat of Reliance Plastic Appliance Conditioner to the plastic surface to be bonded. Wait at least 60 seconds before applying the bonding paste.

BONDING TO A COMPOSITE RESTORATION:

Lightly abrade the composite surface with a fine diamond or microetcher, rinse and dry. Apply a thin uniform coat of Reliance Plastic Conditioner and allow to dry for 60 seconds. Apply a thin coat of bonding resin to the tooth. Proceed with bracket placement.

NOTE: A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C). Avoid excessive heat.

 Flammable  Harmful to Aquatic Life.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

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