PAD LOCK[®]

**FLUORESCENT LIGHT CURE BRACKET
BONDING SYSTEM
WITH OR WITHOUT FLUORIDE**

INSTRUCTIONS FOR USE

RELIANCE[®]

**ORTHODONTIC
PRODUCTS**

RELIANCEORTHODONTICS.COM

V15.3.20 
2797

**FLUORESCENT LIGHT CURE BRACKET BONDING SYSTEM
WITH OR WITHOUT FLUORIDE**

EN ENGLISH TRANSLATION

INDICATIONS FOR USE: Pad Lock® is intended for use as a fluorescing, light cure orthodontic bracket adhesive.

INSTRUCTIONS FOR USE:

Tooth Preparation: Prophylax the enamel surface. Rinse and dry. Etch the enamel surface. Rinse and dry. Apply 1 coat of Light Bond Sealant or Assure® Plus and air dry.

Bracket Application: Using a spatula, smear Pad Lock® paste into the mesh of the bracket pad. Place bracket on the tooth and press in the desired position. Remove flash and light cure.

PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

Etchants are corrosive.

Special Safety Note: Brush applicators and mixing pads are single use devices. It is required that a new brush applicator and mixing pad sheet are used for each patient. Mixing wells can be autoclaved.

TEMPERATURE LIMITATION: Store refrigerated (2-8°C). Avoid excessive heat. Allow product to reach room temperature prior to use.

CONTENTS OF KIT: (4) 4.5g Paste in push syringes or tips, (2) 3cc Light Bond™ sealants, 9g Liquid Etchant, accessory pack.

TO REORDER:

PLP	4.5GM PAD LOCK PASTE- PUSH-NON-FLUORIDE	PLPKFAS	PAD LOCK ECON KIT- PUSH-W/FLUORIDE & ASSURE SEALANT
PLPKAS	PAD LOCK ECON KIT- PUSH-NON-FLUORIDE & ASSURE SEALANT	PLPF	4.5GM PAD LOCK PASTE- PUSH W/FLUORIDE
PLT	4.5GM PAD LOCK PASTE IN TIPS- NON-FLUORIDE	PLTP	4.5 GM PAD LOCK THIN IN PUSH SYRINGE - NON FLUORIDE
PLPKF	PAD LOCK ECONOMY KIT PUSH W/FLUORIDE	PLTF	4.5GM PAD LOCK PASTE IN TIPS W/FLUORIDE

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: Federal law restricts this device to sale or on the order of a dental professional.

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