
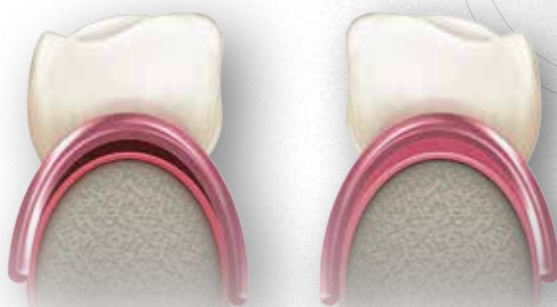


RELINING RESINS

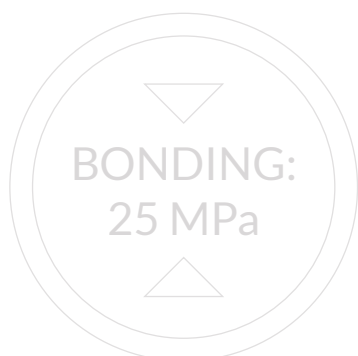

Relines

COMFORTABLE AND LONG-LASTING

in just 1 session



PROCLINIC EXPERT
RELINING MATERIALS

COMPLETE SOLUTIONS

**Proclinic Expert
Soft relining
material**

In tubes
and cartridges

**Proclinic Expert
Hard relining
material**

In cartridges

Total relining	☒	☒
Partial relining	☒	☒
Temporary dentures	☒	☒
Permanent dentures	☒	☒
Improved support and comfort	☒	☒
Denture extensions	☐	☒
Denture edge extensions	☒	☒
Applied in layers	☐	☒
Atrophy	☒	☒
Ulcers	☒	☒
Inflammation	☒	☐
Postoperative	☒	☐

PROCLINIC EXPERT RELINING MATERIALS

The **Proclinic Expert** relining material provides your clients with improved comfort, stability and retention for their dentures. Plus it is quick, easy to use, long-lasting and can be applied in just 1 session.




1
Preparing
the denture



2
Applying
the adhesive



3
Applying
the relining
material

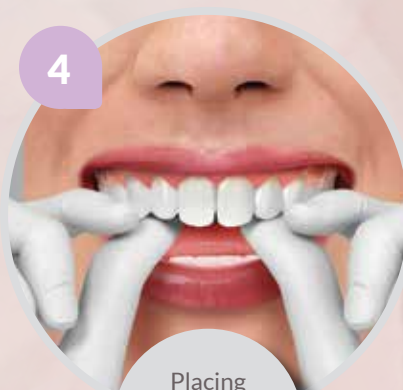

4
Placing
the denture in
the patient's
mouth




5
Varnishing
and polishing

PROCLINIC EXPERT SOFT RELINING MATERIAL

Put an end to the discomfort when chewing

The **Proclinic Expert Soft Relining Material**, made from A-silicone, provides durable elasticity capable of absorbing and redistributing forces. It has long-lasting colour stability and guarantees your patients will benefit from maximum comfort in just 1 session, as well as being odourless and flavourless.

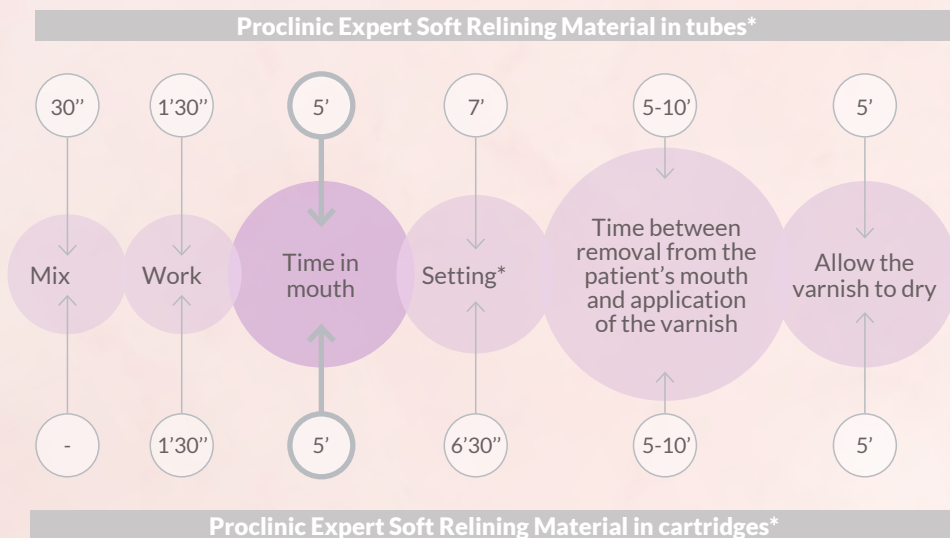

Dr. Vincenzo Dominijanni
Spoleto, Perugia (Italy).

"After trying other relining materials, which after a few weeks start to give off a strong smell, I decided to try Proclinic Expert Soft Relining Material in cartridges, and found it was quick and easy to apply.

Upon removing the denture from the patient's mouth, I discovered a smooth, polished surface, which helped with the final touches. One month after the treatment, the denture was in excellent condition, with a glossy, homogeneous and consistent surface, plus the patient praised the absence of any taste or odours like the ones they noticed with their previous relining materials."



**PROCLINIC EXPERT
HARD RELINING MATERIAL**

Obtain long-lasting results

The **Proclinic Expert Hard Relining Material** in auto-mix cartridges can be applied quickly and easily in just 1 session for complete repairs.

It forms a strong bond to all types of acrylic dentures, ensuring an effective, long-lasting treatment, without any gradual loss of aesthetic qualities or colour stability.

1%

Very low fluid absorption properties

INITIAL COLOUR

AFTER EXPOSURE
TO COFFEE



Sample placed in coffee at 27 °C for 5 days

EXCELLENT COLOUR STABILITY

**VERY LOW
SOLUBILITY**

**BONDING
STRENGTH | 25 MPa**



Proclinic Expert Soft Relining Material in Tubes

- Rapid relining in just 1 session
- Permanently elastic material
- Odourless and flavourless

KIT: 1 x 50ml base tube, 1 x 50 ml catalyst tube, 5 ml adhesive, 2 x 7ml varnish (base + catalyst), 2 pipettes

Ref. 41522

Refill tubes: 50ml base, 50ml catalyst

Ref. 41523

5ml adhesive for tubes

Ref. 41525

Proclinic Expert Soft Relining Material in Cartridges

- Rapid relining in just 1 session
- Permanently elastic material
- Odourless and flavourless

KIT: 50ml cartridge, 5ml adhesive, 2 x 7ml varnish (base + catalyst), 7 tips, 2 pipettes

Ref. 41520

50ml refill cartridge

Ref. 41521

5ml adhesive for cartridges

Ref. 41524



Proclinic Expert Hard Relining Material

- Excellent bonding strength
- Long-lasting repairs
- Durable colour stability

KIT: 50ml cartridge, 15ml adhesive, 6 mixing tips, 6 applicator brushes, 6 dispensing pipettes

Ref. 41526

www.proclinic-products.com

info@proclinic-products.com