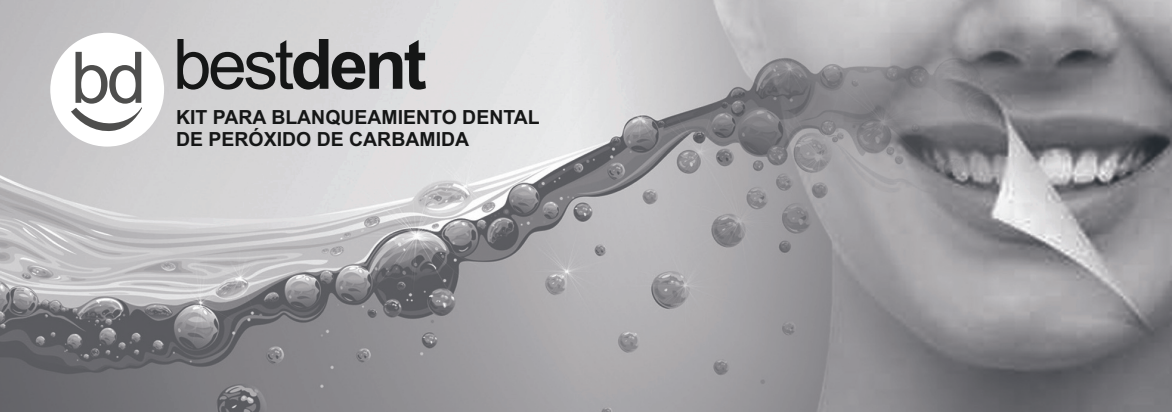
bestdent

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EN **Instructions for use.**
BESTDENT HOME KIT

Content: 3 3 ml syringes of carbamide peroxide whitening gel

- Instructions
- Contains hydrogen peroxide
- Avoid contact with eyes. Rinse immediately if product comes into contact with the eyes.
- Concentration of H₂O₂ present or released as a percentage.

- Do not use in children under 18 years.
- Exclusively for sale to dentists.
- In each cycle of use, the first application must be performed by a dentist or under direct supervision if an equivalent level of safety is ensured. After the first application it may be supplied to the consumer to complete the cycle of use.

INSTRUCTIONS

- Once you have prepared the splint, both the upper and inner arch, pour the product on to the labial surface of the splint (the one which will be in contact with the tooth surface to be whitened).
- Repeat the same procedure with the other splint for the other arch.
- Place the splints without excessive biting. Avoid swallowing the product.
- Do not eat or drink during the process.
- After 30-40 minutes with the splints in place, remove and rinse with water.
- Repeat the process daily until the product runs out. Apply just before bedtime.

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