

StatusBlue™

The Automix Alginate Replacement

■ The faster, easier alternative to alginates



Advantages

- Firm consistency for slump-free tray dispensing.
- Sets quickly for greater patient comfort.
- Stays dimensionally stable.



 **DMG**
A M E R I C A

StatusBlue puts an end to the mess, trouble and lost chair time caused by alginates.

Time To Take Impression* (in minutes)

	Prep time	Time to set in mouth	TOTAL TIME
Alginates	1:45	4:45	6:30
StatusBlue	0:45	1:45	2:30

Time saved with StatusBlue = 4 mins. per impression

DO THE MATH:

If you take an average of 4 impressions a day, the time you save with StatusBlue earns you up to **55 extra hours of valuable chair time each year!**

* Time based on manufacturers' internal testing.



PUSH



PLACE



POUR

There is simply no faster, easier choice for situation impressions.

Top lecturers teach that alginates – with their time-consuming hand-mixing, mess and cleanup – are strictly old school. The ideal choice to create situation impressions for temporaries, model-cast dentures, orthodontic models and opposing jaw models is StatusBlue.

Firm consistency for slump-free tray dispensing.

StatusBlue has an innovative A-silicone formula for a firm consistency and better patient acceptance than alginates. Fast, easy dispensing options include 50ml cartridges for Automix guns and 380ml cartridges for MixStar.

Sets quickly for greater patient comfort.

StatusBlue sets in the mouth in just 1 minute 45 seconds, for better patient comfort. And its unique viscosity allows it to flow better under bite pressure, so it captures exceptional accuracy and detail.

Stays dimensionally stable.

Unlike alginates, StatusBlue impressions stay dimensionally stable and do not have to be kept moist – so there's no pressure to "pour it or lose it." You can pour the model today, tomorrow or next week! You can even cold sterilize StatusBlue impressions and re-pour as needed.

Physical Properties: StatusBlue

Composition	Vinyl polysiloxanes
Consistency	Type 2, Medium Body
Compression set	≤ 0.7%
Base/Catalyst Ratio	4:1
Total Working Time	1:15 minutes
Recommended Time in Mouth	1:45 minutes
Dimensional Change (after 24 hr)	≤ 0.4%
Shore A Hardness	ca. 40
Strain in Compression	≈ 7.0%

Ordering Information:

999826	StatusBlue for Automix Guns Economy Tub (7 +1 FREE 50ml Cartridges, 16 Automix Tips)
999693	StatusBlue MixStar (1-380ml Cartridge and 10 MixStar Tips)
110411	Type 50 Automix Gun (10:1)
999797	StatusBlue Automix Gun Tips (Bag of 25)

For more information, call us at **1-800-662-6383**. To find out more about all of our innovative dental products, visit our website at www.dmg-america.com



DMG America

242 South Dean Street

Englewood, New Jersey 07631

P: 800-662-6383 • 201-894-5505

F: 201-894-0213 • www.dmg-america.com

