bestdent

SILICONA DE ADICIÓN

DE ADDITIONSSILIKON

EN ADDITION SILICONE

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SL ADICIJSKI SILIKON

ET LISANDIGA SILIKOON

DE ADDITIONSSILIKON

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EN ADDITION SILICONE

The Besdent Addition silicone is an addition-cured impression material for dental impression in one or two steps.

It has a high dimension stability and it is odorless and tasteless. It gives you an optimal elastic recovery and high tensile strength.

CHARACTERISTICS

Putty Soft Fast / Putty Soft Extra Fast: The advanced hydrophilic properties give precise and accurate reproduction of details and decrease the risk of inaccuracy due to saliva. It complies with the norm ISO 4823 Type 0 and ADA n° 19 type I (very high viscosity).

Light Fast / Light Extra Fast: The advanced hydrophilic properties give precise and accurate reproduction of details and decrease the risk of inaccuracy due to saliva. It complies with the norm ISO 4823 Type 3 and ADA n° 19 Type I (low viscosity).

Medium Body: The advances hydrophilic properties give precise and accurate reproduction of details and decrease the risk of inaccuracy due to saliva. It complies with the norm ISO 4823 Type 2 and ADA n° 19 Type I (medium viscosity).

Heavy Body Extra Fast: The advances hydrophilic properties give precise and accurate reproduction of details and decrease the risk of inaccuracy due to saliva. Heavy Body Quick is conform at norm ISO 4823 Type 1 and ADA n° 19 Type I (medium viscosity)

MIXING

Putty Soft Fast / Putty Soft Extra Fast: Measure equal amounts of component A and component B. Using hands mix both components until getting a homogeneous color (30 seconds at maximum).

Light Fast / Light Extra Fast: It comes in cartridges and is applied by handgun. Remove the cap of the cartridge. Insert the cartridge in the support. After use, leave the mixing tip in place until new use. Do not replace the cap. For safety before using the cartridge, scrape out the first 2 or 3 cm of product.

Medium Body: It comes in cartridges and is applied by handgun. Remove the cap of the cartridge. Insert the cartridge in the support. After use, leave the mixing tip in place until new use. Do not replace the cap. For safety before using the cartridge, scrape out the first 2 or 3 cm of product.

Heavy Body Extra Fast: Comes in cartridges and is applied by mixing equipment. Remove the cap of the cartridge (rotate it 90°) Insert the cartridge in the support. After use, leave in

place the mixing nib, until the new use. Not replace the cap. For safety, at the beginning of the use cartridge, trough out the first 2 or 3 cm of product.

TAKING THE IMPRESSION

Putty Soft Fast / Putty Soft Extra Fast: Mix identical proportions of base filler and catalyst by hand until a homogeneous colour mixture is obtained material in impression tray. Place the tray containing the material into the mouth. Hold the tray in the position for at least 2,5 minutes for normal version and 1'30" minutes for quick version. Then apply the second silicone, the body light, following the instruction regarding the light product. The suggested temperature is 23° C. Above 23 ° C, the curing time is less than quoted, below that, the curing time could be extended.

Light Fast / Light Extra Fast: Place the product over putty material placed in the impression tray, or directly over the teeth. Place the tray containing the material into the mouth. Hold the tray in the position for at least 2'30" minutes for normal version and 1'30" minutes for quick version. The suggested temperature is 23°C. Above 23°C the curing time is less than quoted, below that, the curing time could be extended.

Medium Body: Place the product in the impression tray, or directly over the teeth. Place the tray containing the material into the mouth. Hold the tray in the position for at least 2'00" minutes. The suggested temperature is 23°C. Above 23°C the curing time is less than quoted, below that, the curing time could be extended.

Heavy Body Extra Fast: Place the Heavy Body Extra Fast in the impression tray, or directly over the teeth. Place the tray containing the material into the mouth. Hold the tray in the position for at least 2'30" minutes. The suggest temperature is 23°C. Above 23°C the curing time is less than quoted, below that, the curing time could be extended.

WARNING

Putty Soft Fast / Putty Soft Extra Fast: The registration, once the silicone putty has been applied, will be finished 30' after removal from the mouth. It could be disinfected in an appropriate solution.

Light Fast / Light Extra Fast: The registration, once the silicone putty has been applied, will be finished 30' after removal from the mouth. It could be disinfected in an appropriate solution.

Medium Body: The registration, will be finished 30' after removal from the mouth. It could be disinfected in an appropriate solution. Use specific mixer for each type of cartridge.

Heavy Body Extra Fast: The registration, will be finished 30' after remove it from the mouth. It could be disinfected in appropriates solution for it.

USER GROUP / PATIENTS

For dental use only!

These materials are for use in the dental field only and must be used according to the Instructions for Use. They should be used in the patient's mouth and only by trained personnel such as dentists, dental assistants, and specially trained personnel. The manufacturer is not liable for any damage that may occur due to its application in other fields or due to improper use. Furthermore, the user is obliged to check the application of the material before use at his own risk.

PRECAUTION FOR HANDLING

It is an addition-cured product, and it can be polluted by poisonous products. Latex examination gloves can interfere with the usual setting characteristics of silicone material impression. Vinyl powder free gloves are acceptable. To avoid the risk of contamination, remove gloves (away from material) and wash hands thoroughly before mixing. Not to be used with polysulphide or polyester based printing materials, nor with condensation materials. The setting of vinylpolysiloxanes is inhibited by latex and nitrile gloves. Do not touch the preparations or retractor threads with latex gloves. It is recommended that professionals wash their hands thoroughly or use vinyl gloves to remove any traces of impurities, especially when mixing putty. Other materials that may inhibit the setting of impression materials include rubber dams, retraction cords and retraction cord liquids. If the practitioner suspects that the preparation has been contaminated, it is recommended to wash and dry the preparation to remove any traces of impurities.

STORAGE

To retain the characteristics of the product, it should be stored at 23°C and used within 36 months.

- Storage temperature: -5°C to +30°C (-4°F to +40°F)

- See expiry date on package.

- Keep away from direct heat sources.

TECHNICAL DATA

PROPERTIES	PUTTY SOFT FAST	PUTTY SOFT EXTRA FAST	LIGHT BODY FAST	LIGHT BODY EXTRA FAST	MEDIUM BODY	HEAVY BODY EXTRA FAST
Aspect	paste	paste	thixotropic fluid	thixotropic fluid	thixotropic fluid	thixotropic fluid
Colour	Blue / yellow	white / blue	white / orange	white / green	white / purple	white / purple
Density (g/cc a 23°C)	1,55	1,55	1,3	1,3	1,4	1,45
Mixing ratio	1:1	1:1	1:1	1:1	1:1	5:1
Mixing time	30"	30"	-	-	-	-
Working time (23°C)	2'	1'30"	2'15'	1'3"	2'15"	1'30"
Total setting time	3'	2'	4'	2'	3'	2'30"
Hardness (after 24h)	60	60	53	48	58	62
Linear retraction	max 0,2%	max 0,2%	max 0,2%	max 0,2%	max 0,2%	max 0,2%
Elastic recovery	min 99,7%	min 99,7%	min 99,7%	min 99,7%	min 99,7%	min 99,7%

NOTICES

In case of contact with the uncured materials with the eyes, rinse with plenty of water. If irritation persists, consult an ophthalmologist. Vinylpolysiloxanes are chemically resistant. Uncured materials may stain clothing. Keep out of reach of children!

CONTRAINDICATIONS

CRONOLAB impression materials are contraindicated if the patient has a known allergy to any of the components. Side effects: none known to date.

ELIMINATION

Impression materials can be disposed of with household waste. Impression material that comes into contact with the patient's mouth must be disposed of in accordance with the applicable country-specific regulations.

NOTIFICATION OBLIGATION

All serious incidents involving this product must be reported immediately to the manufacturer and the competent authorities.

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