OptraGloss® Extra Oral

[en] Instructions for Use

Universal ceramic polishers for extraoral use

Date information prepared:
2023-11-07/web/Rev.1

Manufacturer:
Ivoclar Vivadent AG
Benderstrasse 2
9494 Schaan / Liechtenstein
www.ivoclar.com

ivoclar

English

1 Intended use

Intended purpose

Extraoral polishing of dental ceramic restorations

Use

For dental use only. The product is not a medical device and must not be applied in the patient's mouth.

Description

OptraGloss® Extra Oral consists of a universal polisher for extraoral use, which is suitable for the high-gloss polishing of restorations made of dental ceramics in two steps, and a polishing paste for optional use. The product consists of the following components:

- Diamond pre-polisher (pre-polishing, PP, dark blue) for pre-polishing in the shapes Flame, Disc and Spiral Wheel
- Diamond high-gloss polisher (high-gloss polishing, HP, light blue) for high-gloss polishing in the shapes Flame, Disc and Spiral Wheel
- Diamond polishing paste

2 Application

Polishing of oxide ceramics (e.g. zirconium oxide) and glass ceramics (e.g. lithium disilicate or leucite glass ceramics)

1st step: For pre-polishing, use OptraGloss Extra Oral PP (dark blue) first.

2nd step: For high-gloss polishing, use OptraGloss Extra Oral HP (light blue).

Optionally, use OptraGloss Extra Oral Polishing Paste with a goat hair brush.

Application notes

- Always apply only light contact pressure.
- Optimal rotation speed:
 - for sintered oxide ceramics and glass ceramics: Pre-polishing (PP, dark blue) and high-gloss polishing (HP, light blue) 10,000 rpm to 15,000 rpm
 - for pre-sintered oxide ceramics: Only use the Spiral Wheel shape
- Pre-polishing (PP, dark blue) max. 5,000 rpm
- High-gloss polishing (HP, light blue) max. 10,000 rpm
- Maximum rotation speed: 20,000 rpm
- Always bring the polisher to rotation before you apply it on the surface to be polished.
- Polishing should be performed using light movements to avoid the creation of grooves.
- Ensure adequate evacuation during polishing.
- Polishers may only be used extraorally.
- We recommend using polishers for only one material type to prevent cross-contamination.
- Close OptraGloss Extra Oral Polishing Paste after use.

- OptraGloss Extra Oral Polishing Paste must not be used on pre-sintered oxide ceramics.
- Thoroughly clean the polished restoration with a steam jet or in an ultrasonic bath.
- The Instructions for Use of the material being used must be observed.

3 Safety information

- The current Instructions for Use are available on the website (www.ivoclar.com/eIFU).
- Do not use polishers that have been damaged.
- Use suction equipment and wear protective gloves, goggles and a face mask.
- Use only handpieces that are technically and hygienically impeccable.
- The handpieces used for the instruments must ensure accurate rotation.
- The polishing instrument must be firmly secured in the handpiece.
- Jamming or leveraging should be avoided during rotation to reduce the risk of breaking the instrument.
- Do not leave the polishing instrument rotating in the same spot for too long since this may cause overheating.
- Polishers tend to vibrate if the maximum admissible rotary speed is exceeded. This may lead to damage of the polisher, deformation of the shank and/or fracture of the instrument.
- Excessive contact pressure may lead to damage of the restoration and may accelerate the wear of the polishing instrument.

4 Warnings

The following safety note on OptraGloss Extra Oral Polishing Paste must be observed:

- Causes serious eye irritation.
Wear protective gloves / eye protection / face protection
If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
If eye irritation persists: Get medical advice/attention.
- Observe the Safety Data Sheet (SDS) for OptraGloss Extra Oral Polishing Paste (available at www.ivoclar.com).

5 Disposal information

Remaining stocks must be disposed of according to the corresponding national legal requirements.

6 Shelf life and storage

- Storage temperature OptraGloss Extra Oral Polishing Paste: 2–28 °C (36–82 °F).
- Do not use the product after the indicated expiry date.
- Expiry date: See note on packaging.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

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