LIGHT BOND™

**LIGHT CURE ORTHODONTIC ADHESIVE
WITH OR WITHOUT FLUORIDE**

INSTRUCTIONS FOR USE

RELIANCE®

**ORTHODONTIC
PRODUCTS**

RELIANCEORTHODONTICS.COM

V4.3.20

LIGHT BOND™

LIGHT CURE ORTHODONTIC ADHESIVE
WITH OR WITHOUT FLUORIDE

EN ENGLISH TRANSLATION

INDICATIONS FOR USE: Light Bond is intended for use as a light cure bracket and lingual retainer adhesive.

INSTRUCTIONS FOR USE:

Tooth Preparation: Prophylax the enamel surface. Rinse and dry. Etch the enamel surface. Rinse and dry. Apply 1 coat of Assure® Plus and air dry.

Bracket Application: Using a spatula, smear Light Bond paste into the mesh of the bracket pad. Place bracket on the tooth and press in the desired position. Remove flash and light cure.

CAUTION: Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠ PRECAUTIONS: Unpolymerized resins such as Light Bond Adhesive may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)
Avoid excessive heat. Allow product to reach room temperature prior to use.

☀ IMPORTANT! Do not expose to direct light for any extended period of time as a partial activation may occur. Paste can be applied to brackets in advance but must be shielded from any ambient light (i.e., cover with a box top).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

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