

GoTo™
LIGHT CURE ORTHODONTIC ADHESIVE
CAT# GTP (Push syringe) or CAT# GTT (Tips)

INDICATIONS FOR USE: GoTo™ is intended for use as a light cure bracket adhesive.

BONDS TO METAL, CERAMIC & COMPOSITE SURFACES: GoTo™ will bond to any enamel, porcelain, composite or metal surface, when properly conditioned.

INSTRUCTIONS FOR USE:

PROPHYLAXIS: Using a rotary instrument with a rubber cup or brush, prophylax the enamel surfaces to be bonded with 1st & Final® or plain pumice. Do not use prophylax pastes which contain oil, as they can leave a film that will inhibit the etch. Special attention should be given to the lingual surface and posterior teeth where patient hygiene may be poor and tartar and calculus may be in excess. Rinse thoroughly and dry teeth with oil and moisture-free compressed air.

ETCH: Dispense etching agent onto mixing pad. Etchant may be liquid or gel. Isolate the teeth for etching. With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Do not rub the etching agent on the enamel. Allow 30 seconds for etching (60 for deciduous teeth). Avoid placing etching agent on soft tissue.

RINSE & DRY: Rinse thoroughly with water and spray combination to properly cleanse the enamel surface. Re-isolate and dry teeth thoroughly. The etched area should appear frosty white. If not, re-etch for an additional 20 seconds. Apply 1 coat of Assure or Assure Plus and dry with air. If Pro Seal is used, apply a thin coat and light cure the Pro Seal for 10 seconds per tooth. Proceed with application of the paste. Note: A self-etching primer may be substituted for conventional phosphoric acid etchant. Self-Etching Primer must be dried thoroughly before proceeding with paste.

APPLICATION OF PASTE:

Apply the adhesive and work it into the bracket base. Place bracket on the tooth and press in the desired position. Caution should be taken not to expel all of the paste from beneath a metal bracket. With a metal bracket, position the curing light to shine from the incisal edge and illuminate for 10 seconds and then 10 additional seconds from the gingival, mesial or distal bracket edge. With a ceramic bracket, cure directly through the bracket from the labial surface for 20 seconds. The light director must be placed as


close to the bracket base as possible during curing. Once the paste has been properly cured, an active arch wire can be placed immediately.

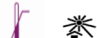
Note: Curing times depend on the intensity of the curing light. The total curing time can range from 6 seconds to 20 seconds. Please refer to the light manufacturer for recommended curing times.

BRACKET REMOVAL:

To remove GoTo™ use a ligature cutter to grab the adhesive at the bracket interface and peel the bracket from the tooth. The majority of the paste will be removed within the bracket base. Remaining material can be removed with the Renew® Finishing System Bur and Renew® Finishing System #383 Point.

CAUTION: Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

 **TEMPERATURE LIMITATION:** Store at room temperature (15-30 Degrees C.) Avoid excessive heat. Do not expose to ambient light. Allow product to reach room temperature prior to use.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Scan for additional information



Reliance Orthodontic Products, Inc.

Australian Sponsor:
Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com