**FICHA TÉCNICA**

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**TECHNICAL DATA SHEET**

|  |   |   |
|--|---|---|
| <b>BESTDENT<br/>BLUE PROTECTIVE FILM</b> |   |  |
| <b>REFERENCES</b>                        | 20980   |   |
| <b>MANUFACTURER</b>                      | CHL MEDICAL SOLUTIONS, Srl<br>Via Felice Cavallotti, 15<br>20122 Milano - Italy   |   |
| <b>CLASSIFICATION</b>                    | Medical Device, Class I, according to Directive 93/42/EEC concerning medical devices.   |   |
| <b>DESCRIPTION</b>                       | Blue adhesive film. The product is non sterile and for single use. Easily removable   |   |
| <b>SPECIFICACIONS</b>                    | 1.200 pre-cut sheets in one roll.<br>Single sheet: 10x15 cm   |   |
| <b>PACKAGING</b>                         | Box with 1 roll   |   |
| <b>INTENDED USE</b>                      | For dental use. Protection of instruments and materials.  |   |
| <b>INSTRUCTIONS FOR USE</b>              | <ul style="list-style-type: none"> <li>• Remove dispensing-slot cover.</li> <li>• Leave the film protector in the box. Insert the end of the film protector into the slot.</li> <li>• Tear off film protector and open its end. Place film protector on turbine and tubing.</li> <li>• Remove and dispose film protector after each patient.</li> </ul> |   |
| <b>SAFETY INFORMATION</b>                | The product can ignite when in contact to a flame. If it burns, it will cause tear-producing smoke that could lead to suffocation.  |   |
| <b>HANDLING AND STORAGE CONDITIONS</b>   | Store in a cool dry place, away from direct sunlight. Protect from moisture.  |   |
| <b>MSDS</b>                              | Not applicable.   |   |