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INSTRUCTIONS FOR USE

EN

PROCLINIC HEMOSTATIC SPONGE MADE OF LYOPHILIZED HYDROLYZED COLLAGEN

INDICATION

Hemostatic Sponge is specially indicated for surgeries because of its high hemostatic and healing power. Hemostatic Sponge should be applied to the cavity where hemostasis is required.

COMPOSITION

Gelatin of pig origin.

DESCRIPTION OF THE PRODUCT

Hemostatic Sponge is a lyophilized gelatin has hemostatic and healing action and it's completely absorbed by the organism within 15 days. It is presented ready for use, individually packaged in blisters.

MECHANISM OF ACTION

Hemostatic Sponge should be applied as an absorbable hemostatic, used to reduce bleeding. Its porous structure allows the product to have a high capacity of fluid absorption. It absorbs at least 35 times its weight of liquid. It is well known that blood clot forms due

to the platelet's agglutination in the place of the bleeding and also due to the activation of the coagulation factors cascade. Hemostatic Sponge works as a matrix that allows additional reinforcement to the formed blood. Besides this local effect, the product has no other pharmacodynamic effects.

INSTRUCTIONS FOR USE

Apply Hemostatic Sponge to the cavity where hemostasis is required. Let's stay in the socket until its complete absorption that happens within 15 days.

CONTRA-INDICATION

None

WARNING

Do not use if pack is damaged. The units must be handled with sterilized tweezers or scissors.

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