IT

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EN

**1. COMPOSITION**

100 g of the solution contains:  
 20 g of iron sulfate (equal to 4.2 g of Fe3+) - hydroglycolic solution q.b.

**2. PROPERTIES**

Receding gums solution with iron sulfate.  
 Astal has a rapid astringent action, with minimal damages to the tissue.

**3. INDICATIONS**

A solution that is designed to have an, immediate temporary and reversible action on the recession of the gums. Making a surgical field free of any blood serum secretions.

Recommended use:

- Before any dental impression
- While performing crown cementation
- While carrying out dental inlays M.O.D. of cervical dental composites or dental amalgams with particular reference to the preparation of cavities with edges beneath the gums.

**4. INSTRUCTION FOR USE**

To eliminate gum secretions and stop small blood losses, one can just brush the gingival sulcus and the surrounding mucosa with ASTAL for a few seconds, followed by a thorough rinse. On the other hand, when a more intense recession of the gums is required before taking a dental impression, do the following:

- When finishing the preparations, brush ASTAL in the gingival sulcus and on the surrounding mucosa for 15/20 sec. then wash thoroughly.
- In preparations with the shoulder, where a precise reading of the edge is required, choose a non-saturated wire of the desired size, gently insert it into the gingival sulcus, and then moisten it with ASTAL using a Penny pipette until it is saturated. Leave ASTAL on for about 15/20 sec., and then rinse thoroughly with water.
- At the end of these procedures the thread can remain in the gingival sulcus until the dental impression is taken.
- In case of persistent bleeding, use ASTAL with a soft bristle brush with a repeating movement; a clot will form which will disappear within 24-48 hours, leaving the tissues clean.

**5. RISKS**

Can be harsh on the mucous membranes if the application times are not respected.

Using ASTAL can cause a light grayish tint of the tissues which is completely harmless and completely reversible in a short time.

**6. PRECAUTIONS**

Do not use with saturated cords with epinephrine, as the latter tints the iron sulfate of a blue color.

**7. WARNINGS**

DO NOT SWALLOW

**Keep out of the reach of children.**

Use protective gloves and glasses.

After use do not dispose the packaging into the environment.

Do not use after the expiration date

Do not use "ASTAL" for any uses other than those indicated in this package leaflet.

**8. CONTRAINDICATIONS**

The components of the product can cause hypersensitivity on an individual basis.

**9. STORAGE**

Keep the bottle tightly closed and away from any heat sources.

**10. PRESENTATION**

15 g. Vial

**11. USAGE REGIME**

For professional use only within the dental practice.

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