IT

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EN

Astringent solution for gingival retraction

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EN

1. COMPOSITION

Aluminium chloride 25%
8-Hydroxyquinoline sulphate and hydroalcoholic excipient (traces)

2. PROPERTIES

The product has considerable astringent power, guaranteeing good local haemostasis and contraction of the surface layers of the gum.

3. INDICATIONS

- in gingival retraction for in situ impregnation of the cord
- bleeding gums

4. INSTRUCTIONS

Haemostatic action: apply 1-2 drops of product using a dropper. Wait 1-2 minutes before thoroughly rinsing the area of application.

Gingival retraction: insert a retraction cord (not medicated) and soak it with a few drops of Aluchlor solution. Leave the cord in place for 3-5 minutes, then remove it, ensuring that no filaments are left in the gingival furrow, then rinse thoroughly.

5. RISKS

Irritant for eyes and skin.

6. PRECAUTIONS

Use protective gloves and glasses.
Do not exceed the application times, in order to prevent the possibility of gum irritation.
Rinse the area of application thoroughly after use to eliminate residues of aluminium salts, which could cause irritation.
Do not use the product with medicated cords.

7. WARNINGS**DO NOT SWALLOW**

Keep out of the reach of children.
Do not dispose of carelessly after use.
Do not use after the expiry indicated on the box.
Do not use ALUCLOR for purposes other than those indicated here.

8. SIDE EFFECTS

The product is not suitable for patients who are sensitive to aluminium or quinoline compounds.

9. STORAGE

Keep the bottle tightly closed and away from sources of heat.

10. PRESENTATION

15 g bottle with dropper.

11. USAGE REGIME

Medical device for use exclusively by specialists in dentistry and stomatology

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