IT

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BIO-ORANGE contains limonene (1,8 -p-menthadiene) in an inert flavored carrier.

2. PROPERTIES

BIO-ORANGE is an organic solvent having similar properties to the common hydrocarbon solvents (xylene, toluene, chloroform). It differs from these because, in addition to having a pleasant aroma, its main ingredient is of plant origin and is much less toxic.

BIO-ORANGE also has good biodegradability indexes and does not present any particular problems for the disposal.

3. INDICATIONS

A solvent used for removing zinc-based cements and dental impression materials from the teeth, hands and all steel instruments and surfaces.

4. INSTRUCTION FOR USE

Dispense the product with the appropriate micro-pump onto a cotton swab and pass it along surfaces which need to be treated.

5. RISKS

The product irritates the eyes and skin.

6. PRECAUTIONS

Avoid prolonged inhalation of the vapors.

7. WARNINGS

DO NOT SWALLOW

Keep out of the reach of children.

Use protective gloves and glasses.

After use do not dispose the packaging into the environment.

Do not use after the expiration date

Do not use "BIO ORANGE SOLVENT" for any uses other than those indicated in this package leaflet.

8. CONTRAINDICATIONS

The components of the product can cause hypersensitivity on an individual basis.

9. STORAGE

Keep the bottle tightly closed and away from any heat sources.

10. PRESENTATION

The package contains 2 bottles each with 200 ml of the product along with a micro-pump.

11. USAGE REGIME

For professional use only within the dental practice

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