IT

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EN

1. COMPOSITION

Mixture of aromatic hydrocarbons and excipients q.s.

2. PROPERTIES

The product has a marked solvent activity towards endodontic cements based on zinc oxide/eugenol (ZOE) and towards gutta percha.

Unlike other solvents already on the market used for these fields of application, ZOE remover contains substances characterised by low toxicity and acts in a few minutes.

3. INDICATIONS

Helps in the removal of apical closures based on endodontic eugenol cements with or without gutta-percha.

4. INSTRUCTIONS FOR USE

After having mechanically cleaned the chamber and the canals, introduce a drop of product and continue the instrumentation phase, taking care to keep the tip of the instruments wet with the product. Continue several times until reaching the apex.

5. RISKS

The product is harmful by ingestion and inhalation. The product is flammable.

6. PRECAUTIONS

Avoid leaving the bottle open in an unventilated environment.

7. WARNINGS**DO NOT SWALLOW**

Use appropriate protective clothing and protect eyes and face. Keep out of reach of children.

Do not dispose of the packaging in the environment after use.

Do not use the product after the expiry date.

Do not use "Zoe Remover" for uses other than those listed in this leaflet.

8. CONTRAINDICATIONS

Individual hypersensitivity to the components of the product.

9. STORAGE

Keep the bottle tightly closed and away from sources of heat.

10. PRESENTATION

15 g bottle with micropipette.

11. USAGE REGIME

Reserved for professional use in dental practice.

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