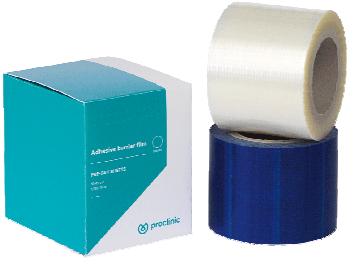
FICHA TÉCNICA

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TECHNICAL DATA SHEET

ADHESIVE BARRIER FILM PROCLINIC		
REFERENCES	49970: blue colour 49971: transparent	
MANUFACTURER	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
CLASSIFICATION	Medical Device, Class I, according to Directive 93/42/EEC	
DESCRIPTION	Adhesive film for protection. The product is non sterile and for single use.	
COMPOSITION	Made of low density polyethylene.	
SPECIFICACIONS	1.200 pre-cut sheets in one roll. Single sheet: 10x15 cm Thickness: 42 µm ±2 Adhesive: Acrilic in water suspension Adhesivity: >10 N/cm	
PACKAGING	Box with 1 roll	
SHELF LIFE	3 years from the date of manufacture	
INTENDED USE	For dental use. Protection of sleeves, panels, lights, air/water syringes.	
INSTRUCTIONS FOR USE	<ul style="list-style-type: none"> • Remove dispensing-slot cover. • Leave the film protector in the box. Insert the end of the film protector into the slot. • Tear off film protector and open its end. Place film protector on turbine and tubing. • Remove and dispose film protector after each patient. 	
SAFETY INFORMATION	<p>Good tolerance to temperature, oily substances, UV rays and ageing. Easily removable.</p> <p>The product can ignite when in contact to a flame. If it burns, it will cause tear-producing smoke that could lead to suffocation.</p>	
HANDLING AND STORAGE	Store in a cool dry place, away from direct sunlight. Protect from moisture.	