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TDS_051_1

Date: 20/Apr/17

ROOT CANAL SEALER PROCLINIC EXPERT		
REFERENCES	78550	
MANUFACTURER	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
CLASSIFICATION	Medical Device, Class IIa, Directive 93/42/EEC. Notified Body: Kiwa Cermet Italia S.p.A., No. 0476	
DESCRIPTION	Two-paste epoxy-amine resin root canal sealer in a dual auto-mixing syringe.	
COMPOSITION	Paste 1: Bisphenol-A-(epichlorhydrin) and epoxy resin, Zirconium Oxide, Silica, Iron Oxide pigments. Paste 2: Poly(oxyethylene, oxypropylene)diamine, Silica, Zirconium Oxide, Antibacterial agent.	
PACKAGING	5 ml Automix Syringe, 10 Automix Syringe Mix Tips and Inta Oral Tips, 1 Mixing Pad.	
SHELF LIFE	2 years	
INTENDED USE	Permanent obturations of root canals combined with root canal points.	
INSTRUCTIONS FOR USE	<p>Prepare the root canal, clean and dry before use of Proclinic Expert Root Canal Sealer.</p> <p>Remove the cap by turning it 90° counterclockwise and pulling it off. Bleed the syringe prior to use.</p> <p>Attach the mixing tip to the syringe and then turn the tip 90° clockwise. Auto mixing syringe ensures proper homogeneous mixing of two pastes and allows direct application of the sealer into the root canal.</p> <p>After the application, the mixing tip must be removed by rotating it 90° counter clockwise and pulling it off. Then the auto mixing syringe must be closed with the original cap.</p>	

CHL MEDICAL SOLUTIONS, Srl**TECHNICAL DATA SHEET**

TDS_051_1

Date: 20/Apr/17

PRECAUTIONS & CONTRAINDICATIONS	<p>Hypersensitivity to epoxy resins and amino groups. Attention: do not to use in case of patient who has allergenic sensitivity to epoxies and amines.</p> <p>In case of contact with eyes or skin: wash immediately with plenty of water. Any excess of material must be removed with cotton applicator and then washed with water followed with suction.</p> <p>For external use only! If swallowed, drink plenty of water. If any sickness symptoms seek medical assistance.</p>
HANDLING AND STORAGE CONDITIONS	<p>To be stored at temperature from 4 to 24°C.</p> <p>The syringe must be closed after use with original cap only.</p>

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CHL MEDICAL SOLUTIONS, Srl**FICHA TÉCNICA**