KALCIUMHYDROXIDPASTA MED RÖNTGENKONTRAST

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CALCIUM HYDROXIDE PASTE WITH RADIOCAPACITY

INDICATIONS

1. Temporary root filling material for conventional root treatment.
2. Pulp capping.
3. Perforation and pulp protection.
4. Insulation in deep cavities.

ADVANTAGES

- Stimulates formation of dentine bridges.
- Has strong bactericidal effect.
- Stimulates hard tissue formation in roots.
- Quick and simple application of calcium hydroxide.
- Eliminates mixing of powder and liquid.
- Always the ideal consistency.

CONTENTS

Calcium hydroxide 41%,
Barium sulphate 8%,
Sterile isotonic saline solution
pH 12.4

DIRECTIONS FOR USE

1. Before application eject a small amount of paste - to equalise pressure in the syringe and to control flow of paste through the needle.
2. For a temporary root filling apply paste gradually and intermittently compress with absorbing material like paper points or cotton pellets. In very narrow root canals applicat paste by spiral root filling instrument (Lentulo).
3. Each syringe can be used several times. To guarantee perfect function the needle should be discarded and replaced with the security cap after use.

All contact between calcium hydroxide paste and air will result in formation of calcium carbonate which will harden after some time.

ADDITIONAL NOTES

- Proclonic Expert Calcium Hydroxide Paste is corrosive.
- Skin contact: Wash with plenty of water. Remove splashed clothes and wash underlying skin.
- Eye contact: Immediately rinse with copious amounts of water and seek medical advice.
- Ingestion: Drink water or preferably milk (0,5 liter), and seek medical advice.

SHELF LIFE

Two years from date of manufacture.

STORAGE

Store at room temperature, do not refrigerate. Recommended storage < 25°C. Avoid exposure to direct sunlight.

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DIRECTA AB