Instructions for use

EN

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DE

#### Universal Ultrasonic Cleaner Concentrate

Enzyme cleaner designed to remove a wide range of contaminants. Designed for use with BioSonic® Ultrasonic cleaning units. Ideal for dental instruments and equipment.

- Contains anti-rust inhibitors
- Aldehyde free
- Phosphate free
- Non ammoniated
- Biodegradable.

#### Directions for Use:

Add 20ml of the concentrate to 1 Litre of warm water (35-40 °C) in the ultrasonic bath (2% solution). Immerse the instruments in the freshly prepared cleaning solution. Ensure all surfaces are covered by the solution; add hollow objects in such a way that air escapes. Run the ultrasonic cleaner for a minimum of 15 minutes and according to the manufacturer’s instructions. After immersion, remove the instruments and rinse them thoroughly prior to terminal sterilization or high level disinfection. Prepare a fresh solution daily or sooner if solution is very dirty.

#### Danger:

H332-Harmful if inhaled. H318-Causes serious eye damage. P101-If medical advice is needed, have product container or label at hand. P102-Keep out of reach of children. P261-Avoid breathing dust/fume/gas/mist/vapours/spray. P280-Wear protective gloves/protective clothing/eye protection/face protection. P304+P340-IF INHALED:Remove person to fresh air and keep comfortable for breathing.P305+P351+P338-IF IN EYES:Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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**Topdental (Products) Ltd.** 🏢
**Coltene/Whealed GmbH + Co. KG**