
VELOCE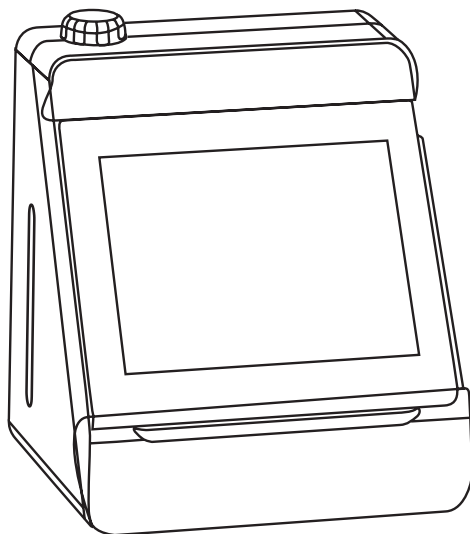

LUBRICATION MACHINE VELOCARE

EN_ Instructions for use

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FOREWORD

For the safe and effective operation of this device, please read this user manual carefully before using the product. The device is used for lubricating dental handpieces. The device is suitable for use in dental clinics and hospitals. The user manual gives detailed instructions on installation, operation, maintenance, and troubleshooting of the device.

USER RECORDS

You will find the product name, model number and serial number from the label which is stuck to the back side of the device. Please check and record the serial number in the blank space below. Please advise the serial number when you try to contact your local distributor.

Product Name: LUBRICATION MACHINE VELOCARE

Model Number: LUB-120

Serial Number: _____

CONTENTS

I. Basic Characteristics.....	4
II. Parameter.....	5
III. Accessory Name.....	5
IV. Installation Of Components.....	6
V. Operation Steps.....	9
VI. Maintenance.....	13
VII. Troubleshooting.....	15
VIII. Warranty.....	15

Our company is responsible for device's safety, reliability, and performance, within the following conditions:

1. The device's installation, debugging and after service work should be done by professional personnel approved by our company or personnel who have the corresponding national technical qualification.
2. The power supply of the device should comply with related national regulations and device's usage requirements.
3. The device's operation should be done by qualified or approved personnel, the operator should have the professional medical application skills and operate based on this user manual.

SAFETY WARNING

1. Please read below safety clauses before you correctly use and operate this handpiece lubricating machine LUB-120.
2. The following instructions can guide you on how to use the device safely, avoid harm to you and other people caused by improper operation. All risk remarks are indicated according to diverse levels of danger:

Danger Precautions	to remind you about the improper operations which may result in death and serious injury
Warning Precautions	to remind you about the improper operation which may result in physical injury and machine damages.
Safety Precautions	to remind you about the improper operation which may result in physical injury and machine damages

DANGER PRECAUTIONS

1. Prohibit installing, using, and filling the product with oil in explosive spaces or in places close to fire sources.
2. Prohibit installing and using the device under direct sunlight or in environments where the temperature exceeds 40 °C (104 °F).
3. Prohibit using the device for purposes not mentioned in this user manual, except for cleaning and lubricating. Please keep the device in a well-ventilated environment regularly. If there is a peculiar smell inside the device, ensure proper ventilation.

SAFETY PRECAUTIONS

1. Do not touch the power cable with wet hands to avoid electric shock.
2. Do not pour water onto the machine to avoid electric shock caused by fire, disaster, or short circuit.
3. Do not disassemble or change the device. Please contact the local distributor if you need after-sales service.
4. If there is smoke or smell of burning resin coming from the machine, please turn off the electricity and unplug the socket immediately, and at the same time contact the local distributor.
5. Avoid dropping the device. Always place it on a stable, horizontal surface.
6. After use, please turn off the power and cut off the air supply.
7. The air supply (such as an air compressor) and air filter supplied to the machine should be kept clean and dry. The moisture in the air compressor and air filter should be drained regularly because the moisture in the cavity mixed with the maintenance oil will cause harm to the machine.
8. If the equipment will be left without use for a long time, please remove the air pipe and power cord.
9. When storing and maintaining lubricating oil, please keep it away from oxidants, flammable substances, and strong acid chemicals as much as possible.
10. Please purchase substitutes specified by our company, such as lubricating oil, etc. Any unspecified parts may cause damage and malfunction of the machine.

I. BASIC CHARACTERISTICS

1. Ensure the cleaning and oiling of the handpiece.
2. Available to clean and oil three handpieces at the same time.
3. The rotating gear in our handpiece can effectively clean and lubricate from different angles.
4. After completing the cleaning and lubricating steps, pressing the air compartment can remove the excess residue in the handpiece.
5. Choose short mode, long mode, and extra-long mode to lubricate according to different handpieces.
6. Oil mist filter can prevent the diffusion of oil mist to a minimum.

II. PARAMETER

1. Model: LUB-120.
2. Rated voltage: 100V ~ 240V ~; 50/60.
3. Output power: 60VA.
4. Working pressure: 0.4 ~ 0.6MPa (4.0 ~ 6.0kgf/cm²) (60 ~ 80 psi).
5. Oil drum capacity: 300 ml.
6. Size: (mm): 310 × 270 × 350.

III. ACCESSORY NAME

1. The names of parts are shown in Figure 1 and Figure 2:

Figure 1

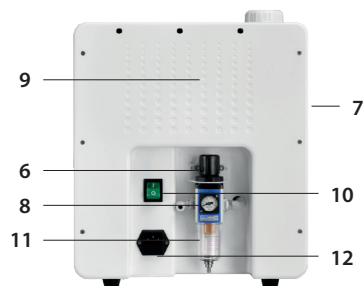
Figure 2

- | | |
|---|-------------------|
| 1. Oil cap | 7. Pressure gauge |
| 2. Operational panel | 8. Air filter |
| 3. Door cover | 9. Beep hole |
| 4. Oil level gauge | 10. Switch |
| 5. Shell | 11. Power socket |
| 6. Operation pressure adjustment button | 12. Fuse box |

Accessory

The following accessories are equipped with each machine. If anyone is missing, please contact your dealer:

1. Lubrication oil _____ 1 bottle
2. Oil-absorbing cotton _____ 1 bag (15 pieces)
3. Oil funnel _____ 1 piece
4. Power cord _____ 1 set
5. Y-shaped connector _____ 1 piece
6. O-ring _____ 4 pieces
7. Air pipe _____ 1 piece

IV. INSTALLATION OF COMPONENTS

1. Rated Fuse

Check the voltage to ensure it is consistent with the rated fuse according to the chart below:

Voltage	Rated fuse
100V-240V	T3A/250V

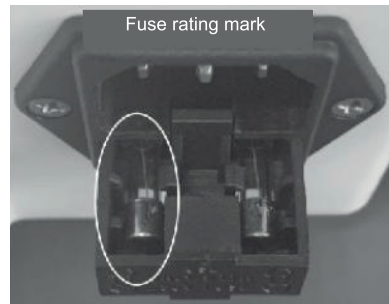
Figure 3

2. Connect The Air Tube

Insert one end of the air tube into the interface of the air filter and connect the other end directly to the air compressor for connection. If you need to connect other medical equipment, please use the attached Y-type connector to connect.

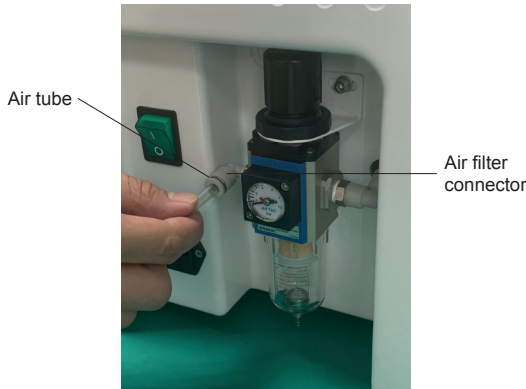
Figure 4

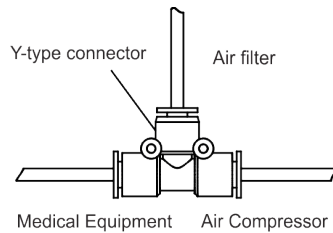
Figure 5

⚠ Warning Precautions

1. Working pressure ranges from 0.4–0.6MPa (4.0–6.0kgf/cm²) (60–80 psi)
2. Working pressure is no less than 0.4MPa (3.5kgf/cm²) (50 psi)
3. Do not wind or bend the air tube.
4. If you cannot install the air tube, please contact the supplier.
5. The air source (like air compressor) and air filter which supplies the machine should be kept clean and dry. Regularly drain moisture from the air compressor and air filter because the moisture inside the chamber will be mixed with the lubricating oil, which will do harm to the machine.

3. Add Oil / Oil Drainage

(1) Add Oil

Turn on the machine, rotate the oil cap, put the oil funnel into the refuelling port and pour in the lubricating oil. Stop refuelling when the oil level reaches the "FULL" position, then remove the oil funnel and tighten the oil cap.

Figure 6

Attention:

Please add the lubricating oil in time when the oil level is below the "EMPTY" position.

⚠ Warning Precautions

1. Before refuelling, please check that there is no clutter in the refuelling funnel to prevent blockage of the lubrication system.
2. Pay attention to the bottom of the machine drain bolts: whether they are tightened. Bolts not tightened will lead to lubricating oil leakage.
3. Be sure to refuel at start-up.
4. To avoid oil leakage, please pour the oil slowly during refuelling. Do not exceed the "FULL" position.
5. Make sure the cap is screwed tight after refuelling.
6. Not using the machine for a long time or in the process of transportation, please empty the oil storage tank.
7. Do not add the lubricating oil in a space where there is a danger of explosion and near the source of fire.

4. Connect The Power

Please insert the power supply into the socket on the back of the machine.

Figure 7

V. OPERATION STEPS

1. How to install several types of handpieces

- Installation of high-speed pneumatic turbine handpiece

(Based on 1S09168 standard)

1. Remove the needle from the handpiece
2. Open the door cover, install the handpiece to the connector and tighten the nut (As shown in Figure 8)
3. Close the door carefully.

Figure 8

⚠ Safety Precautions: Be sure to remove the needle before installing the handpiece on the connector of the lubrication device.

- Installation of low-speed handpieces (based on 1S03964 standard)

Straight handpieces

1. Open the door cover, (as shown in Figure 9)

Install the handpiece into the low-speed connector until the handpiece and the interface are locked.

2. Close the door carefully.

Angled handpiece

1. Remove the needle from the handpiece.

2. Open the door cover, (as shown in Figure 9)

Install the handpiece into the low-speed connector until the handpiece and the interface are locked.

3. When installing, "please turn the pin jack of the angled handpiece towards the door cover, and carefully close the door cover (as shown in Figure 10)

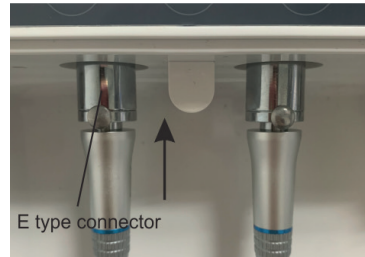
Figure 9

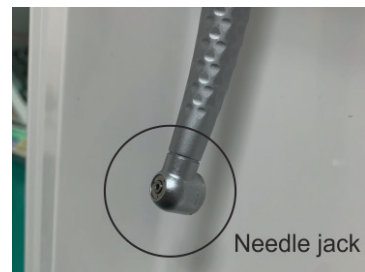
Figure 10

⚠ Safety Precautions

1. Make sure the needle has been removed before installing the handpiece into connector, and you can also install a standard clamping rod for maintenance.
Before installing the right-angle handpiece, the needle needs to be locked.
2. The pin jack of the angled handpiece needs to face the door cover. If the direction is wrong, the door cover will not close.
3. When installing the high-speed manual turbo handpiece, be sure to insert to the connector in place, and then tighten the nut to avoid damage to the handpiece or nut thread.
4. After the installation of the low-speed handpiece, please gently pull the handpiece to make sure it is installed in place.

2. Cleaning And Lubrication Steps

1. After turning on the power and turning on the power switch, the power indicator lights (see Figure 11).

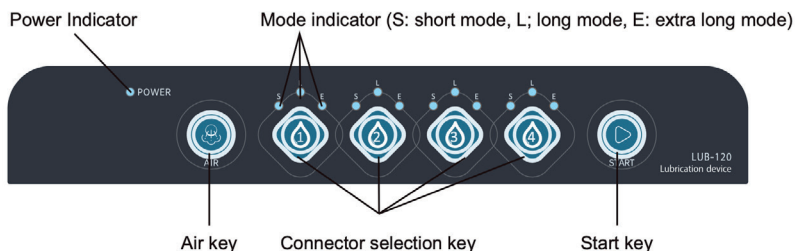
Figure 11

2. Select the 1st, 2nd, and 3rd keys according to the handpiece installed in the corresponding connector. Press once to select "S" for short mode, press twice to select "L" mode for long mode, and press three times for "E" for extra-long mode. These modes are indicated by S, L, and E lights.
3. If you have two or more handpieces, please select the correct mode for each handpiece.
4. Press the start button, the machine will automatically start the cleaning and oil filling procedures. During the operation of the machine, the indicator lights of different modes will keep flashing.
5. After the machine finishes cleaning and filling the handpiece, the mode indicator will stop flashing, and the buzzer will give an audible prompt.
6. Press the air button, at this time, air will blow out air from the joint interface to remove excess oil left in the handpiece.

Mode	Clean oil filling time	Air cleaning time
Short mode	About 35s	About 60s
Long mode	About 45s	About 80s
Extra long mode	About 55s	About 90s

Safety Precautions

1. Use the correct power supply: 100V-240V ~
2. Select the corresponding mode according to several types of handpieces. The correct mode can clean the oil from the handpiece. .
3. When the mode indicator is not lit, the machine will not operate normally.
4. When the mode indicator is not lit, the machine will not operate normally.

Icon	Meaning
!	About 35s
⚠	See attached files
🧑	Class B equipment
SN	Product Serial Number
○	Off (disconnected from the power supply)
I	On (connected to the power supply)

Handpiece Disassembly Steps

1. Disassembly of gas turbine handpiece (According to the standard ISO9168):

Mode indicator lights up again, and after the cleaning and lubrication process is completed, open the cover (As fig 12 shows). Then loosen the nut and pull out the handpiece from the connection.

Figure 12

2. Disassembly of low-speed handpiece (According to the standard ISO3964):

Mode indicator lights up again, and after the cleaning and lubrication process is completed, open the cover (As fig 13 shows). Then loosen the nut and pull out the handpiece from the connection.

Figure 13

⚠ Safety Precautions

The lubricant remaining on the bottom of the door cover will leak. Use a cotton cloth to wipe off the lubricating oil in the bottom and the door cover. Dispose of the cotton cloth as medical waste according to regulations.

VI. MAINTENANCE

1. Replacement of oil mist filter cotton and oil-absorbing cotton. (As fig 14 shows)

Safety Precautions

Replacement of oil mist filter cotton and oil-absorbing cotton after about 50 uses.

Deal with these cotton that belong to medical waste products according to the law.

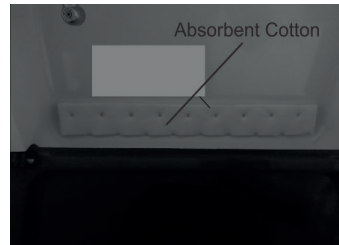
Figure 14

2. O-ring replacement on low-speed handpiece connection

Use your fingers to take out the O-ring and put the new O-ring into the groove. (As fig 15 shows)

Figure 15

3. Replacement Of Air Filter

1. Remove the air tube from the connection. (As fig 16 shows)
2. Take out the air filter after removing the screw. (As fig 17 shows)
3. Install a new filter and insert the air tube into the connection as much as possible.

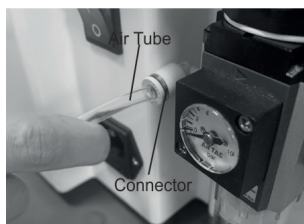
Figure 16

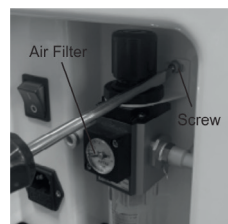
Figure 17

Notes:

- Only remove the filter when the air source is completely closed.
- Do not install the filter in reverse.
- Expected service life: 8 years.

4. Replacement of air filter components, as shown in figure 18-19

Figure 18

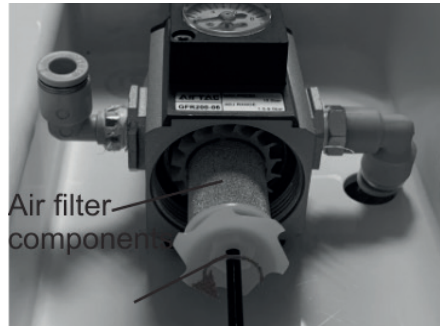
Figure 19

Air filter for drainage, as shown in figure 20.
Replacement of fuse, as shown in figure 21.

Figure 20

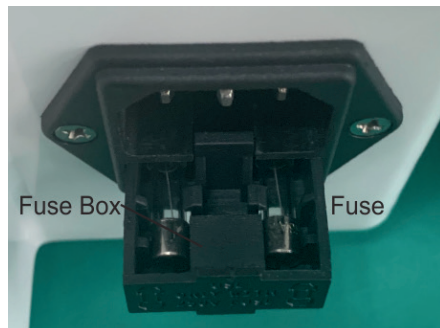
Figure 21

VII. TROUBLE SHOOTING

If the machine is faulty, refer to the table below for similar conditions before maintenance is required. If there is no corresponding situation, contact your agent for repair.

Situation	Possible Reason	Troubleshooting
No power	The power cord is not connected to the external power supply	Check external power
	Whether the power cord is plugged into the power interface of the machine	Check whether the power cord and the machine power plug are secure
	Whether the power switch is on	Check power switch
	Whether the fuse is blown	Check fuse
The machine cannot be cleaned and oiled	Whether the mode indicator is on	Press the mode key
	Whether the air pipe is connected	Check the trachea for air pressure
	Are handpieces installed	Check handpieces

VIII. WARRANTY

The product has defects in materials and workmanship during normal installation, use and maintenance, the manufacturer gives the buyer quality assurance, the maintenance oil, filter cotton, O-ring and other components that are consumables are not covered by the warranty. If the product fails within 30 days after installation. Please contact the dealer immediately and show the purchase invoice.

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PROCLINIC S.A.U.
Ningbo Runyes Medical Instrument Co., Ltd.