Teeth whitening kit

35% HYDROGEN PEROXIDE

Instructions for use

- EN** Instructions for use.
- ES** Instrucciones de uso.
- FR** Mode d'emploi.
- IT** Istruzioni per l'uso.
- PT** Instruções de uso.
- PL** Instrukcja postępowania.

INSTRUCTIONS FOR USE

IN-OFFICE WHITENING KIT 35% HYDROGEN PEROXIDE

Contents:

- 3 double syringes 2 ml. 35% hydrogen peroxide
- 3 protective resins x 1.2 g
- 3 polishing pastes x 1.2 g
- Application tips

It contains hydrogen peroxide, xylitol and aloe vera. – Adverse effects if ingested. – It irritates the skin. – Avoid ingestion. – Avoid contact with the skin. – Avoid contact with the eyes. – In case of contact with eyes, rinse immediately with plenty of water and consult a doctor. Not intended for use by individuals under 18 years old. – Sale exclusively for odontologists. It is not recommended to regular consumers of tobacco or alcohol. – Do not use on children or pregnant women.

Instructions:

1. Carry out oral prophylaxis prior to the treatment.
2. Check the teeth and their initial shade before the application of the treatment using the color guide in the following order: B1, A1, B2, D2, A2, C1, C2, D3, A3, D4, B3, A3.5, B4, C3, A4, C4.
3. Registry the initial shade with photographs. Adjust a lip retractor. Use the polishing paste included in the kit to remove possible stains. Never use polishing pastes containing glycerin, oils or rubber.
4. Dry the teeth and the gums with compressed air and then apply the gum protection covering completely all the gingival tissue. Cure the resin with the light. Caution: overexposure of the resin to the lamp may cause an increase of the heat in the gums.
5. Apply a 2 mm layer of the product using the syringe in the front face of the teeth.
6. If you are using a light, place the lamp at 2 cm from the lips. Adjust the time and leave the product for 8-10 minutes.

7. If you are NOT using a whitening lamp, start with a 10-minute cycle and apply the curing light at 6-7mm from the surface of the tooth, changing to a different tooth each 3 seconds until finishing the 10 minutes of the cycle.
8. Once completed the first cycle, remove the gel carefully from each tooth. Apply the product again and use the light. It is not necessary to remove the protective resin, as it will be useful for the whole treatment.
9. Once completed the second cycle, remove the product again. If you still haven't obtained the desired results, you can apply a third cycle exactly like the previous ones. Apply a maximum of 3 cycles of a duration of 10 minutes each.
10. Rinse off and remove the gum protection.
11. Finally, you can apply diamond paste as a finishing touch.
12. Check again the shade of the teeth and take another photograph to compare it to the initial shade.
13. During the next 72 hours the patient should avoid the ingestion of the following: tobacco, coffee, tea, cola, red wine, grape juice, artichoke, beets, cherries, strawberries, blackberries, raspberries, pomegranate, watermelon, black grapes, chocolate, cocoa, carrots, broccoli, spinach, tomato sauce, squid ink, stews containing coloring or any other food with an intense coloring.
14. It is recommended to use in combination with the Proclinic Expert Home Use whitening treatment during 1 week.

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PROCLINIC, S.A.U.