

## Color-Coded Sizes

NEODRYS are color coded for quick identification. Available in two sizes: **Large** (blue) and **Small** (yellow). Packaged in convenient, easy-to-handle packs of 50.

### Small (Yellow)

- Original Backing
- Reflective Backing



## Choice of Backing

Available in Reflective light-enhancing, mirror-like finish or Original white backing.

### Large (Blue)


- Original Backing
- Reflective Backing




## Method of Application

Select NEODRYS size which adequately covers the buccal mucosa. Insert as shown with color side against cheek and point to back of mouth. In a few seconds, NEODRYS will begin to adhere to the tissue and stay in place. **Important:** To remove without irritating tissue, release adhesion with ample water spray to the buccal side of the NEODRYS.


## Applying NeoDrys

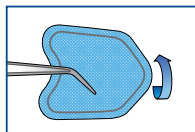

COLOR TO CHEEK

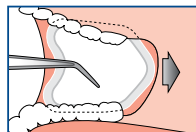

POINT TO BACK

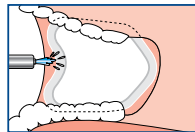

SPRAY TO RELEASE



Dental Advisor's  
5+ Highest Rating



Dentaltown's  
Townie Choice Award

Made in the U.S.A.  
US7425664  
EP1676552

Distributed by:



3120 Moon Station Road, NW  
Kennesaw, GA 30144, U.S.A.  
sales@microcopydental.com  
www.microcopyintl.com



Prima Dental Group  
Stephenson Drive  
Gloucester  
GL2 2AG, UK



DY 0615

# NEODRYS®

## Ultimate Parotid Saliva Control

Absorb, Protect & Reflect!



“NEODRYS have no substitute - making them an item you simply cannot work without.”

James J. Corbett III DDS, Baton Rouge, LA

MICROCOPYINTL.COM





## ABSORB, PROTECT & REFLECT.

Stop wasting money on pads and rolls that either fall apart or get saturated quickly. When you add up the reduction in botched procedures and wasted time — *NEODRYS cost less!*



*"NEODRYS do not fall apart like the other absorbents and I can usually do a whole procedure without having to switch them out. Usually I would have to use 2-3 of the other absorbents. Great product!"*

L. Bell, CDA

*"I really like these NEODRYS! They work well with CEREC® AC."*

Alex Touchstone, DDS  
Certified CEREC Trainer

*"NEODRYS Reflective is one of the absolute best inventions in the field of dentistry. The addition of the reflective surface substantially brightens the oral cavity."*

Edna Sheffield, RDH

### ABSORB

- ◆ Super absorbent core—out performs messy cotton rolls and other absorbent pads.
- ◆ No messy leaks, even with the core fully saturated.



### PROTECT

- ◆ Protects procedures from moisture contamination from the parotid gland for up to 15 minutes.
- ◆ Stays in place yet removes easily with water spray.
- ◆ Protective backing plus poly stiffener provide cheek retraction and protection.

### REFLECT

- ◆ Choice of Original white or Reflective backing brightens the oral cavity, while retracting the cheek for an improved view.

## NEODRYS FEATURES

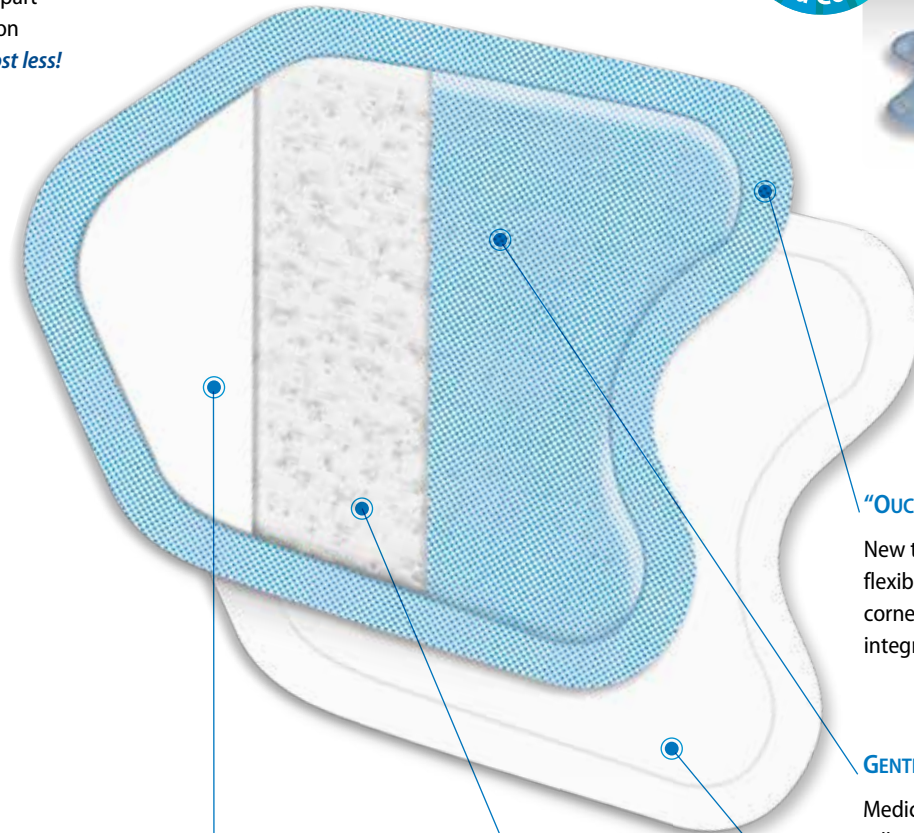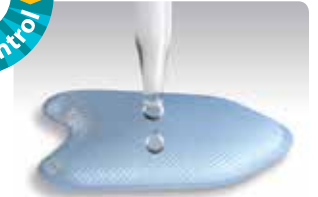

### "OUCHLESS" COMFORT EDGE®

New technology creates a soft, flexible edge with no sharp corners while retaining product integrity.

### GENTLE TO THE BUCCAL MUCOSA

Medical-grade poly netting adheres gently to tissue.

### CORE STIFFENER

Poly stiffener insert provides optimum rigidity for cheek retraction.

### SUPER ABSORBENT CORE

Acrylate polymer traps moisture as a gel when in contact with saliva.

### PROTECTIVE BACKING

Choice of Original white or Reflective backing brightens the oral cavity while providing cheek protection.