FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT STERILIZATION HEAT SEALING ROLL		
REFERENCES	79944: 50 mm x 200 m 79945: 75 mm x 200 m 79946: 100 mm x 200 m	
MANUFACTURER	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
CLASSIFICATION	Medical Device, Class I, according to Directive 93/42/EEC.	
DESCRIPTION	Sterilization rolls for single use	
COMPOSITION	Made of medical use laminated transparent film and medical paper.	
PRODUCT SPECIFICACIONES	Sterilization with steam: the indicator colour changes from blue to black. Sterilization with ethylene oxide: the indicator colour changes from pink to yellow.	
PACKAGING	Box with one sterilization roll	
SHELF LIFE	5 years from the date of manufacture	
INTENDED USE	Sterilization of medical devices and materials for medical use which are especially suitable for steam or ethylene oxide gas.	
SAFETY INFORMATION	Place the pouch with the transparent side downwards (in contact with the tray) and the paper side upwards. The objects to be wrapped must be placed in an orderly way at the centre, leaving about 1 cm of space between the objects and the heat sealing on the sides. The objects must furthermore be placed lengthwise with respect to the lateral heat sealing of the rolls, avoiding that any part pushes on the heat seals themselves. If these limits cannot be respected, use a larger sterilization roll.	
HANDLING AND STORAGE CONDITIONS	Store in a cool dry place, away from direct sunlight. Protect from moisture. Do not use if the package is damaged.	
STANDARDS APPLIED	ISO 11607-1 EN 868-2,-3,-5	

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