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EN SILICONE FOR BITE REGISTRATION

VINILPOLYSILOXANO HIGH HYDROCOMPATIBILITY FOR BITE REGISTRATION

Dental use product. Keep out reach of children

User group

Product for use in dentistry, only to be used by trained personnel and according to the instructions for use. The manufacturer is not liable for damage caused by improper use or use in other areas. Keep out of the reach of children.

Presentation

2 cartridges of 50 ml + 12 mixer

Characteristics

Bestdent Bite Registration is one addition-cured impression material for bite registration. Bestdent Bite Registration is extremely hard for taking precise non-deformable occlusal registration. Bestdent Bite Registration is Conform to the norm ISO 4823 Type 1.

TECHNICAL DATA	
Aspect	thixotropic fluid
Color	blue
Density (g/cc a 23°C)	1,70
Mixing ratio	1:1
Working time (23°C)	20 seconds
Total setting time (beginning until the end)	1 minute
Hardness (after 24h)	80 Shore A
Linear retraction	maxi 0,2%
Elastic recovery	more 99,7%

Advantage

Bestdent Bite Registration has a high dimension stability and it is odorless and tasteless. Bestdent Bite Registration gives you an optimal elastic recovery and high tensile strength.

Cartridge preparation

Remove the cap of the cartridge (rotate it 90°). Insert the cartridge in the support. After use, leave in place the mixing nib, until the new use. Not replace the cap. For safety, at the beginning of the use cartridge, trough out the first 2 or 3 cm of product.

Use

Bestdent Bite Registration, it is presented in cartridges and it is applies by a hand gun. It can be put in a tray or placed directly on the teeth. Place the tray containing the material into the mouth. Hold the tray in the position at least 1'. The suggest temperature is 23°C. Above this temperature the suggested time is less than quoted, below, the time is extended.

Warning

The bite registration will be finished 30' after remove it from the mouth. It could be disinfected in appropriates solution for it. Use the mixer specifics for every cartridge.

Precautions for handling

Bestdent Bite Registration, is one addition-cured product, and can be polluted by poisoners products. Latex examination gloves can interfere with the usual setting characteristics of silicone material impression. Vinyl Power-free gloves are acceptable. To avoid the risk of contamination, remove gloves (away from material) and wash hands thoroughly before mixing.

Product storage

To retain the characteristics of Bestdent Bite Registration should be stored at 23°C and used within 36 months.

Disposal

Impression materials can be disposed of with household waste. Impression material that comes into contact with the patient's mouth must be disposed of in accordance with country-specific regulations.

Notification Obligation

All serious incidents involving this product must be reported immediately to the manufacturer and the competent authorities.

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